



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

| | |
|---|----------------------------------|
| Serial Number: <u>80-004950</u> | Instrument Location: <u>BURK</u> |
| Reason for Install/Repair: | |
| <input type="checkbox"/> Install After Lab Repair/Inspection <input checked="" type="checkbox"/> Other (Specify) <u>NEW INSTRUMENT</u> | |

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000. N/A
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

[Signature]
 Field Inspector Signature

4-23-2010
 Date

Deb Kashur
 Reviewed By

10 May 2010
 Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BURK 8164.13.00 06/09
04/23/2010 13:25

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 2
Standard Expiration? 12/23/2011
Oper No? 049303

Flow Cal. Date: 01/18/2010
Slope 774
Intercept -908904

IR Calibration Date: 02/11/2010
 3um 9um

0th Coef(*100): -29762 -38464
1st Coef(*100): 275393 145322
2nd Coef(*100): 383 366
H2O adj(mg/l*10k): 725 627

***** Printer Test End *****

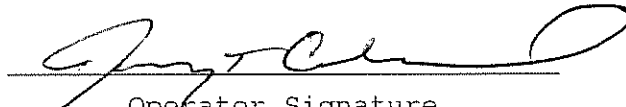

Operator Signature
JEREMY T. GANDRUD

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BURK 8164.13.00 06/09
04/23/2010 13:47

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 13:48 |
| 02 Std. Gas | 0.078 | 13:48 |
| 03 Room Air | 0.000 | 13:49 |
| 04 Std. Gas | 0.079 | 13:49 |
| 05 Room Air | 0.000 | 13:50 |
| 06 Std. Gas | 0.078 | 13:50 |
| 07 Room Air | 0.000 | 13:50 |

Lot No = 709352
Cyl No = 2
Exp Date = 12/23/2011
County = 07 Oper No. = 049303

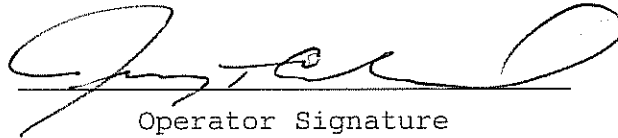

Operator Signature
JEREMY T. GANDRUD

Remarks:

Form 106-I8000

80-004950
P. 47/48 JAG

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BURK 8164.13.00 06/09
04/23/2010 13:57

| Test | AC | Time |
|---------------|-------|-------|
| 01 Diagnostic | OK | 13:59 |
| 02 Room Air | RFI* | 13:59 |
| 03 Room Air | 0.000 | 13:59 |

*Invalid Test
Inhibited - RFI

Sub Name = GANDRUD, JEREMY TODD
Sub DOB = 07/03/1969
Sub Sex = Male Weight = 180
Test = OTH Cit = NA
Dr. Lic. = ND/GAN000000
Lot No = 709352
Cyl No = 2
Expiration Date = 12/23/2011
County = 07 Oper No. = 049303

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

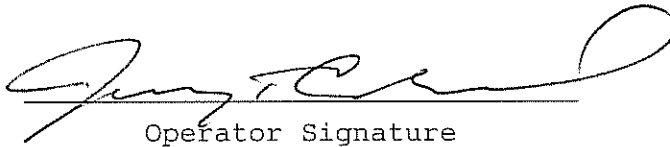

Operator Signature
JEREMY T. GANDRUD

Remarks:

Form 106-I8000

80-004950
P. 48/4P FM