

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003057

Location: WASH - McLean Co

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
  - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3846 Lot No. 1652 Rep AC 0.10 Ac + 0.05% acetone ✓
  - H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 54 Exp. Date 08/01/2014  
Test 1 0.079 Test 4 0.078 Test 7 0.079  
Test 2 0.079 Test 5 0.079 Test 8 0.079  
Test 3 0.079 Test 6 0.079 Test 9 0.078  
Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 957 psi Reg 970 psi

K. Remarks/Maintenance record: annual inspection

24 Apr 2014  
Date

24 APR 2014  
Date

Debo Kasheer  
Field Inspector's Signature

Charles E. Edrington  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003057  
Location = WASH                8164.13.00 06/09  
04/24/2014                      13:43

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            54  
Standard Expiration?       08/01/2014  
Oper No?                    777777

Flow Cal. Date:            11/29/2010  
Slope                      653  
Intercept                  -334316

IR Calibration Date:       04/02/2013

	3um	9um
0th Coef(*100):	-13529	-8119
1st Coef(*100):	242413	133015
2nd Coef(*100):	5279	1746
H2O adj(mg/l*10k):	635	475

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = WASH      8164.13.00 06/09  
04/24/2014      14:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:04
02 Std. Sol.	0.021	14:05
03 Room Air	0.000	14:05
04 Std. Sol.	0.021	14:06
05 Room Air	0.000	14:07
06 Std. Sol.	0.021	14:07
07 Room Air	0.000	14:08

08 Sim Temp = 34.0°C

Simul Ser No = DR5115  
Std Sol No = 12111  
County = 28      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

Low AC      0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = WASH      8164.13.00 06/09  
04/24/2014      13:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:59
02 Std. Sol.	0.302	13:59
03 Room Air	0.000	14:00
04 Std. Sol.	0.303	14:01
05 Room Air	0.000	14:01
06 Std. Sol.	0.303	14:02
07 Room Air	0.000	14:02

08 Sim Temp = 34.0°C

Simul Ser No = DR5143  
Std Sol No = 13080  
County = 28      Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*High AC 0.300 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = WASH      8164.13.00 06/09  
04/24/2014      13:44

Test	AC	Time
01 Room Air	0.000	13:44
02 *Subject Test	INT*	13:45
03 Room Air	0.000	13:46

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 54  
Expiration Date = 08/01/2014  
County = 28      Oper No. = 777777

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

*Interferent check 0.10AC + 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003057  
Location = WASH                    8164.13.00 06/09  
04/24/2014                            13:46

Test	AC	Time
01 Diagnostic	OK	13:47
02 Room Air	RFI*	13:47
03 Room Air	0.000	13:48

\*Invalid Test  
Inhibited - RFI


Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 54  
Expiration Date = 08/01/2014  
County = 28                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB KASHUR

Remarks:



Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = WASH      8164.13.00 06/09  
04/24/2014      13:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:51
02 Std. Gas	0.079	13:52
03 Room Air	0.000	13:52
04 Std. Gas	0.079	13:52
05 Room Air	0.000	13:53
06 Std. Gas	0.079	13:53
07 Room Air	0.000	13:54

Lot No = 16512080A2  
Cyl No = 54  
Exp Date = 08/01/2014  
County = 28      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = WASH      8164.13.00 06/09  
04/24/2014      13:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:55
02 Std. Gas	0.078	13:55
03 Room Air	0.000	13:56
04 Std. Gas	0.079	13:56
05 Room Air	0.000	13:56
06 Std. Gas	0.079	13:57
07 Room Air	0.000	13:57

Lot No = 16512080A2  
Cyl No = 54  
Exp Date = 08/01/2014  
County = 28      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = WASH      8164.13.00 06/09  
04/24/2014      14:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:12
02 Std. Gas	0.079	14:12
03 Room Air	0.000	14:13
04 Std. Gas	0.079	14:13
05 Room Air	0.000	14:13
06 Std. Gas	0.078	14:14
07 Room Air	0.000	14:14

Lot No = 16512080A2

Cyl No = 54

Exp Date = 08/01/2014

County = 28

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080AC*

Form 106-I8000