

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004948

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>12210</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR384</u> Lot No. <u>13020</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR384</u> Lot No. <u> </u> Rep AC <u>0.100 AC + acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>23411080 A1</u>	Cyl No. <u>15</u>	Exp. Date <u>10/01/2013</u>
Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 371 psi Reg 400 psi

M. Remarks/Maintenance record: Annual inspection, replaced breath hose o ring & external battery.

24 Apr 2013
Date

Deb Kashur
Field Inspector's Signature

25 APR 2013
Date

Charles E. Ehr
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = ~~STUP~~ ^{TOXL} 8164.13.00 06/09
04/24/2013 ^{DK} 09:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:19
02 Std. Sol.	0.021	09:20
03 Room Air	0.000	09:21
04 Std. Sol.	0.020	09:21
05 Room Air	0.000	09:22
06 Std. Sol.	0.020	09:23
07 Room Air	0.000	09:23


08 Sim Temp = 34.0°C

Simul Ser No = DR5132

Std Sol No = 12210

County = 08

Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = ~~STUT TOXL~~ 8164.13.00 06/09
04/24/2013 DK 09:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:12
02 Std. Sol.	0.299	09:12
03 Room Air	0.000	09:13
04 Std. Sol.	0.299	09:14
05 Room Air	0.000	09:14
06 Std. Sol.	0.299	09:15
07 Room Air	0.000	09:15

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 13020
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
04/24/2013 10:42

Test	AC	Time
01 Diagnostic	OK	10:42
02 Room Air	RFI*	10:42
03 Room Air	0.000	10:43

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
04/24/2013 09:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:27
02 Std. Gas	0.080	09:28
03 Room Air	0.000	09:28
04 Std. Gas	0.080	09:28
05 Room Air	0.000	09:29
06 Std. Gas	0.080	09:29
07 Room Air	0.000	09:30

Lot No = 23411080A1

Cyl No = 15

Exp Date = 10/01/2013

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = TOXL 8164.13.00 06/09
04/24/2013 09:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:50
02 Std. Gas	0.081	09:51
03 Room Air	0.000	09:51
04 Std. Gas	0.080	09:52
05 Room Air	0.000	09:52
06 Std. Gas	0.080	09:52
07 Room Air	0.000	09:53

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

