



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (5-2011)

| | | |
|---|---|---------------------------------------|
| Chemical Test Operator Name (Print) <i>Daniel Blood</i> | | |
| Location <i>Watford City Police Dept.</i> | Intoxilyzer® Serial Number <i>80-005946</i> | |
| Gas Lot Number <i>16512080A2</i> | Gas Cylinder Number <i>64</i> | Gas Expiration Date <i>08/01/2014</i> |

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.077 AC
 B. 0.077 AC
 C. 0.077 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

| | |
|--|---------------------------|
| Chemical Test Operator Signature <i>Daniel Blood</i> | Date <i>04-24-2014</i> |
| Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i> | Date <i>25 Apr 2014 ✓</i> |


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WCPD 8164.13.00 06/09
04/24/2014 11:40

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 11:40 |
| 02 Std. Gas | 0.077 | 11:41 |
| 03 Room Air | 0.000 | 11:41 |
| 04 Std. Gas | 0.077 | 11:41 |
| 05 Room Air | 0.000 | 11:42 |
| 06 Std. Gas | 0.077 | 11:42 |
| 07 Room Air | 0.000 | 11:43 |

Lot No = 16512080A2
Cyl No = 64
Exp Date = 08/01/2014
County = 27 Oper No. = 120701



Operator Signature
DANIEL BLOOD

Remarks: *ACA Test*