

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005362

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5114 Lot No. 2016034 Rep AC 0.010 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. JCS4 Solution 0.10AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 1421408046 Cyl No. 053 Exp. Date 7/5/2016<sup>6</sup><sub>CFE</sub>

Test 1	<u>0.078</u>	Test 4	<u>0.078</u>	Test 7	<u>0.079</u>
Test 2	<u>0.078</u>	Test 5	<u>0.078</u>	Test 8	<u>0.079</u>
Test 3	<u>0.078</u>	Test 6	<u>0.078</u>	Test 9	<u>0.079</u>

Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 206 psi Reg 200 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED SIMULATOR

RETURN O-RING

4.20.2016  
Date

25 Apr 16  
Date

Charles E. Ede  
Field Inspector's Signature

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer          Alcohol Analyzer  
North Dakota Model 8000          SN 80-005362  
Location = TOXL                    8164.13.00 06/09  
04/20/2016                                  10:46

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

Current Instrument Setup

Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                    1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    14214080A6  
Standard Cyl #?                    53  
Standard Expiration?                07/05/2016  
Oper No?                              666666  
  
Flow Cal. Date:                    08/19/2011  
Slope                                667  
Intercept                            -577009

IR Calibration Date:                04/07/2015  
    3um                                  9um

-----  
0th Coef(\*100):                    -22003                              -17935  
1st Coef(\*100):                    254570                              134851  
2nd Coef(\*100):                    4320                                2032  
H2O adj(mg/l\*10k):                466                                 435

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005362  
Location = TOXL      8164.13.00 06/09  
04/20/2016      11:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:01
02 Std. Sol.	0.010	11:02
03 Room Air	0.000	11:03
04 Std. Sol.	0.010	11:03
05 Room Air	0.000	11:04
06 Std. Sol.	0.010	11:04
07 Room Air	0.000	11:05

08 Sim Temp = 34.0°C

Simul Ser No = DR5114  
Std Sol No = 201603H  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.010 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005362  
Location = TOXL      8164.13.00 06/09  
04/20/2016      11:06

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:07
02 Std. Sol.	0.407	11:08
03 Room Air	0.000	11:08
04 Std. Sol.	0.408	11:09
05 Room Air	0.000	11:09
06 Std. Sol.	0.407	11:10
07 Room Air	0.000	11:11

08 Sim Temp = 34.0°C

Simul Ser No = MP3068  
Std Sol No = 14102  
County = 08      Oper No. = 666666

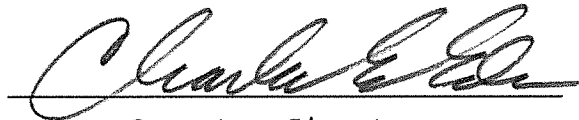

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005362  
Location = TOXL                    8164.13.00 06/09  
04/20/2016                            11:13

Test	AC	Time
01 Room Air	0.000	11:14
02 *Subject Test	INT*	11:14
03 Room Air	0.000	11:15

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 53  
Expiration Date = 07/05/2016  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005362  
Location = TOXL                    8164.13.00 06/09  
04/20/2016                            11:15

Test	AC	Time
01 Diagnostic	OK	11:16
02 Room Air	0.000	11:16
03 *Subject Test	RFI*	11:17
04 Room Air	0.000	11:17

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 53  
Expiration Date = 07/05/2016  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

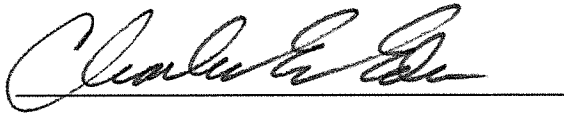

Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005362  
Location = TOXL      8164.13.00 06/09  
04/20/2016      11:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:18
02 Std. Gas	0.078	11:18
03 Room Air	0.000	11:19
04 Std. Gas	0.078	11:19
05 Room Air	0.000	11:20
06 Std. Gas	0.078	11:20
07 Room Air	0.000	11:20

Lot No = 14214080A6  
Cyl No = 53  
Exp Date = 07/05/2016  
County = 08      Oper No. = 666666

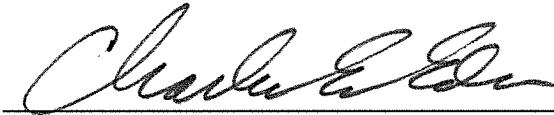

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005362  
Location = TOXL      8164.13.00 06/09  
04/20/2016      11:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:22
02 Std. Gas	0.078	11:22
03 Room Air	0.000	11:23
04 Std. Gas	0.078	11:23
05 Room Air	0.000	11:23
06 Std. Gas	0.078	11:24
07 Room Air	0.000	11:24

Lot No = 14214080A6  
Cyl No = 53  
Exp Date = 07/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005362  
Location = TOXL      8164.13.00 06/09  
04/20/2016      11:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.079	11:25
03 Room Air	0.000	11:26
04 Std. Gas	0.079	11:26
05 Room Air	0.000	11:27
06 Std. Gas	0.079	11:27
07 Room Air	0.000	11:28

Lot No = 14214080A6  
Cyl No = 53  
Exp Date = 07/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000