

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005939

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3058 Lot No. 201501A Rep AC 0.015 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 19615080A5 Cyl No. 003 Exp. Date 9/5/2017

|                     |                     |                     |
|---------------------|---------------------|---------------------|
| Test 1 <u>0.081</u> | Test 4 <u>0.082</u> | Test 7 <u>0.082</u> |
| Test 2 <u>0.082</u> | Test 5 <u>0.082</u> | Test 8 <u>0.082</u> |
| Test 3 <u>0.082</u> | Test 6 <u>0.082</u> | Test 9 <u>0.082</u> |

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 495 psi Reg ~500 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED BREATH HOSE O-RING, COULD NOT GET BAR CODE TAPED TO RN PROTECTIVE AREA ON BREATH HOSE, SPANNER TO FAIL DURING TESTING

4.21.2016  
Date  
25 Apr 16  
Date

Cleaveland E. Ed  
Field Inspector's Signature  
Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:03

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890- = |  
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@# \$%^&\* ( ) \_ + ?

abcdefghijklmnopqrstuvwxy1234567890- = |  
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@# \$%^&\* ( ) \_ + ?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      19615080A5  
Standard Cyl #?      3  
Standard Expiration?      09/05/2017  
Oper No?      666666

Flow Cal. Date:      06/18/2013  
Slope      676  
Intercept      -672473

IR Calibration Date:      06/18/2013

|                    | 3um    | 9um    |
|--------------------|--------|--------|
| 0th Coef(*100):    | -16377 | -17793 |
| 1st Coef(*100):    | 286199 | 137970 |
| 2nd Coef(*100):    | 3159   | 1297   |
| H2O adj(mg/l*10k): | 522    | 398    |

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:11

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 16:12 |
| 02 Std. Sol. | 0.015 | 16:13 |
| 03 Room Air  | 0.000 | 16:13 |
| 04 Std. Sol. | 0.015 | 16:14 |
| 05 Room Air  | 0.000 | 16:14 |
| 06 Std. Sol. | 0.015 | 16:15 |
| 07 Room Air  | 0.000 | 16:16 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3058  
Std Sol No = 201501A  
County = 08      Oper No. = 666666

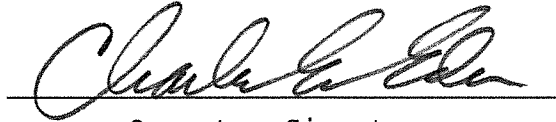

Operator Signature  
CHARLES EDER

Remarks:

LOW AC  
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:17

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 16:18 |
| 02 Std. Sol. | 0.403 | 16:18 |
| 03 Room Air  | 0.000 | 16:19 |
| 04 Std. Sol. | 0.403 | 16:20 |
| 05 Room Air  | 0.000 | 16:20 |
| 06 Std. Sol. | 0.404 | 16:21 |
| 07 Room Air  | 0.000 | 16:21 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3068  
Std Sol No = 14102  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: HIGH AC  
0.400 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:23

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 16:24 |
| 02 *Subject Test | INT*  | 16:25 |
| 03 Room Air      | 0.000 | 16:25 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19615080A5  
Cyl No = 3  
Expiration Date = 09/05/2017  
County = 08      Oper No. = 666666

  
Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:26

| Test          | AC    | Time  |
|---------------|-------|-------|
| 01 Diagnostic | OK    | 16:26 |
| 02 Room Air   | RFI*  | 16:27 |
| 03 Room Air   | 0.000 | 16:27 |

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 19615080A5  
Cyl No = 3  
Expiration Date = 09/05/2017  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:27

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:28 |
| 02 Std. Gas | 0.081 | 16:28 |
| 03 Room Air | 0.000 | 16:29 |
| 04 Std. Gas | 0.082 | 16:29 |
| 05 Room Air | 0.000 | 16:30 |
| 06 Std. Gas | 0.082 | 16:30 |
| 07 Room Air | 0.000 | 16:30 |

Lot No = 19615080A5  
Cyl No = 3  
Exp Date = 09/05/2017  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:31

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:31 |
| 02 Std. Gas | 0.082 | 16:32 |
| 03 Room Air | 0.000 | 16:32 |
| 04 Std. Gas | 0.082 | 16:32 |
| 05 Room Air | 0.000 | 16:33 |
| 06 Std. Gas | 0.082 | 16:33 |
| 07 Room Air | 0.000 | 16:34 |

Lot No = 19615080A5  
Cyl No = 3  
Exp Date = 09/05/2017  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.13.00 06/09  
04/21/2016      16:34

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:35 |
| 02 Std. Gas | 0.082 | 16:35 |
| 03 Room Air | 0.000 | 16:35 |
| 04 Std. Gas | 0.082 | 16:36 |
| 05 Room Air | 0.000 | 16:36 |
| 06 Std. Gas | 0.082 | 16:37 |
| 07 Room Air | 0.000 | 16:37 |

Lot No = 19615080A5  
Cyl No = 3  
Exp Date = 09/05/2017  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000