

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005358

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3066 Lot No. 15160 Rep AC 0.026 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS 4 Solution 0.10AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC  
14214080A6  
Lot No. ~~29614080A6~~ Cyl No. 053 Exp. Date 7/5/2016  
Test 1 0.078 Test 4 0.078 Test 7 0.078  
Test 2 0.078 Test 5 0.079 Test 8 0.078  
Test 3 0.078 Test 6 0.078 Test 9 0.079  
Average 0.078
  - J. Tank pressure, Level 3 Func D Sub G P: 317 psi Reg 300 psi
  - K. Remarks/Maintenance record: ANNUAL INSPECTION

4.18.2016  
Date

25 Apr 16  
Date

Chanda E. Edwards  
Field Inspector's Signature

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005358  
Location = TOXL                    8164.13.00 06/09  
04/18/2016                            16:00

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            14214080A6  
Standard Cyl #?            53  
Standard Expiration?        07/05/2016  
Oper No?                    666666

Flow Cal. Date:            04/17/2014  
Slope                        622  
Intercept                    -324577

IR Calibration Date:        04/17/2015  
                                  3um                    9um

-----  
0th Coef(\*100):            -16867                -25778  
1st Coef(\*100):            266890                139601  
2nd Coef(\*100):            3270                    1479  
H2O adj(mg/l\*10k):        332                    443

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:02
02 Std. Sol.	0.018	16:03
03 Room Air	0.000	16:03
04 Std. Sol.	0.018	16:04
05 Room Air	0.000	16:05
06 Std. Sol.	0.018	16:05
07 Room Air	0.000	16:06

08 Sim Temp = 34.0°C

Simul Ser No = MP3066  
Std Sol No = 15160  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Low AC 0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:09
02 Std. Sol.	0.401	16:10
03 Room Air	0.000	16:11
04 Std. Sol.	0.405	16:12
05 Room Air	0.000	16:12
06 Std. Sol.	0.405	16:13
07 Room Air	0.000	16:14

08 Sim Temp = 34.0°C

Simul Ser No = MP3068  
Std Sol No = 14102  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:15

Test	AC	Time
01 Room Air	0.000	16:16
02 *Subject Test	INT*	16:16
03 Room Air	0.000	16:17

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 53  
Expiration Date = 07/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:17

Test	AC	Time
01 Diagnostic	OK	16:18
02 Room Air	0.000	16:19
03 *Subject Test	RFI*	16:19
04 Room Air	0.000	16:19

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 53  
Expiration Date = 07/05/2016  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:20
02 Std. Gas	0.078	16:20
03 Room Air	0.000	16:21
04 Std. Gas	0.078	16:21
05 Room Air	0.000	16:22
06 Std. Gas	0.078	16:22
07 Room Air	0.000	16:23

Lot No = 14214080A6  
Cyl No = 53  
Exp Date = 07/05/2016  
County = 08      Oper No. = 666666

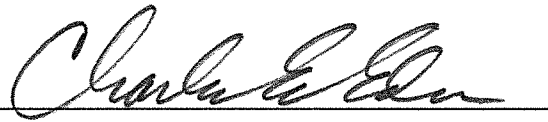

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:24
02 Std. Gas	0.078	16:24
03 Room Air	0.000	16:25
04 Std. Gas	0.079	16:25
05 Room Air	0.000	16:26
06 Std. Gas	0.078	16:26
07 Room Air	0.000	16:27

Lot No = 14214080A6  
Cyl No = 53  
Exp Date = 07/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005358  
Location = TOXL      8164.13.00 06/09  
04/18/2016      16:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:31
02 Std. Gas	0.078	16:32
03 Room Air	0.000	16:32
04 Std. Gas	0.078	16:33
05 Room Air	0.000	16:33
06 Std. Gas	0.079	16:34
07 Room Air	0.000	16:34

Lot No = 14214080A6  
Cyl No = 53  
Exp Date = 07/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
CEE

0.080 AC  
Form 106-I8000