



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-002669</i>	Instrument Location <i>Watford City</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Dwight B Newman #030924</i>	Date <i>04/18/12 21 Apr 2012</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>26 Apr 2012</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = WATF 8164.13.00 06/09
04/18/2012 16:21

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 11611080A2
Standard Cyl #? 114
Standard Expiration? 06/01/2013
Oper No? 030924

Flow Cal. Date: 09/09/2011
Slope 662
Intercept -510765

IR Calibration Date: 09/09/2011
 3um 9um

0th Coef(*100): -15623 -9015
1st Coef(*100): 263092 133867
2nd Coef(*100): 3130 1285
H2O adj(mg/l*10k): 642 392

***** Printer Test End *****

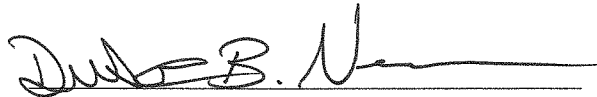

Operator Signature
DUSTIN B NEWMAN

Remarks: *Good Print Test*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = WATF 8164.13.00 06/09
04/18/2012 17:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:01
02 Std. Gas	0.082	17:02
03 Room Air	0.000	17:02
04 Std. Gas	0.082	17:03
05 Room Air	0.000	17:03
06 Std. Gas	0.082	17:04
07 Room Air	0.000	17:04

Lot No = 11611080A2
Cyl No = 114
Exp Date = 06/01/2013
County = 27 Oper No. = 030924

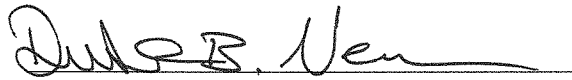

Operator Signature
DUSTIN B. NEWMAN

Remarks: Good ACT

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = WATF 8164.13.00 06/09
04/18/2012 17:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	17:11
02 Room Air	RFI*	17:12

*Invalid Test
Inhibited - RFI

Lot No = 11611080A2
Cyl No = 114
Exp Date = 06/01/2013
County = 27 Oper No. = 030924

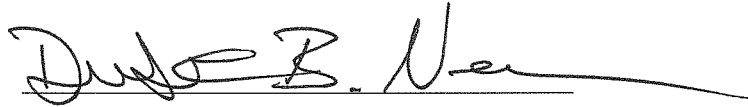

Operator Signature
DUSTIN B. NEWMAN

Remarks:

Good
RFI Test for installation

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = WATF 8164.13.00 06/09
04/21/2012 18:45

Test	AC	Time
01 Diagnostic	OK	18:46
02 Room Air	RFI*	18:46
03 Room Air	RFI*	18:46

*Invalid Test
 Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 03/03/1988
Sub Sex = Male Weight = 180
Test = OTH Cit = 000000
Dr. Lic. = ND/DAK000000
Lot No = 11611080A2
Cyl No = 114
Expiration Date = 06/01/2013
County = 27 Oper No. = 030924

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DUSTIN B. NEWMAN

Remarks: *Good RFI Test*

20 April 2012

I let Dustin Newman know that I didn't an RFI run in a different mode for a proper Intoxilyzer 8000 installation. The RFI he send to the Crime Lab was run in ACA mode, which shows that the RFI is working properly, but I still wanted him to run it according to the Intoxilyzer 8000 Installation and Repair Checkout form in CMS mode.

A handwritten signature in cursive script that reads "Deb Kashur".

Deb Kashur