

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003067

Location: Morton Co jail

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR3803 Lot No. 15160 Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5190 Lot No. ICS4 Solution 0.10AC + 0.05% acetone ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 29614080 A2 Cyl No. 32 Exp. Date 12/05/2016  
Test 1 0.079 Test 4 0.080 Test 7 0.079  
Test 2 0.079 Test 5 0.079 Test 8 0.079  
Test 3 0.078 Test 6 0.078 Test 9 0.079  
Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 951 psi Reg 970 psi

K. Remarks/Maintenance record: Annual inspection, replaced  
DS simulator breath hose o-ring.

26 Apr 2016  
Date  
4-26-2016  
Date

Deb Shanaver  
Field Inspector's Signature  
Chad Edwards  
Reviewed by

Intoxilyzer Test Record and Checklist  
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer          Alcohol Analyzer  
 North Dakota Model 8000          SN 80-003067  
 Location = MAND                  8164.13.00 06/09  
 04/26/2016                          14:28

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwx yz1234567890- = |  
 ABCDEFGHIJKLMNOPQRSTU VWXYZ!@#\$%^&\*()\_ +?

abcdefghijklmnopqrstuvwx yz1234567890- = |  
 ABCDEFGHIJKLMNOPQRSTU VWXYZ!@#\$%^&\*()\_ +?

Current Instrument Setup

Data Entry Mode:                  Enabled  
 Start Test Sequence:              DABACABA  
 Display Prelim Rslt?              Yes  
 Display Third Digit?              Yes  
 Inhib Printer(Y/N)?              No  
 Display Volume?                  No  
 Disable On Memfull?              Yes  
 # of Print Copies?                1  
 Select Std (D/W/I)?              Dry  
 Standard Value?                  0.080  
 Standard Lot #?                  29614080A2  
 Standard Cyl #?                  32  
 Standard Expiration?              12/05/2016  
 Oper No?                          777777

Flow Cal. Date:                  01/23/2013  
 Slope                              664  
 Intercept                         -509770

IR Calibration Date:              04/30/2015  
     3um                                  9um

-----  
 0th Coef(\*100):                  -19633                              -12115  
 1st Coef(\*100):                  278224                              137001  
 2nd Coef(\*100):                  2596                                 1415  
 H2O adj(mg/l\*10k):              323                                  247

\*\*\*\*\* Printer Test End \*\*\*\*\*

*Deb Shanaver*  
 \_\_\_\_\_  
 Operator Signature  
 DEB SHANAVER

Remarks:

*Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = MAND      8164.13.00 06/09  
04/26/2016      14:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Sol.	0.021	14:43
03 Room Air	0.000	14:44
04 Std. Sol.	0.021	14:45
05 Room Air	0.000	14:45
06 Std. Sol.	0.020	14:46
07 Room Air	0.000	14:46

08 Sim Temp = 34.0°C

Simul Ser No = DR3803

Std Sol No = 15160

County = 30

Oper No. = 777777

*Deb Shanaver*

Operator Signature

DEB SHANAVER

Remarks:

*Low AC 0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = MAND      8164.13.00 06/09  
04/26/2016      14:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:48
02 Std. Sol.	0.312	14:49
03 Room Air	0.000	14:50
04 Std. Sol.	0.309	14:50
05 Room Air	0.000	14:51
06 Std. Sol.	0.308	14:52
07 Room Air	0.000	14:52

08 Sim Temp = 34.0°C

Simul Ser No = DR3847

Std Sol No = 16060

County = 30

Oper No. = 777777

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

High AC      0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = MAND      8164.13.00 06/09  
04/26/2016      14:54

Test	AC	Time
01 Room Air	0.000	14:55
02 *Subject Test	INT*	14:56
03 Room Air	0.000	14:57

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 32  
Expiration Date = 12/05/2016  
County = 30      Oper No. = 777777

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

*Interferent check 0.10 AC + 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

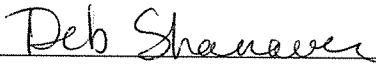
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = MAND      8164.13.00 06/09  
04/26/2016      14:58

Test	AC	Time
01 Diagnostic	OK	14:59
02 Room Air	RFI*	14:59
03 Room Air	0.000	15:00

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 29614080A2  
Cyl No = 32  
Expiration Date = 12/05/2016  
County = 30      Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
DEB SHANAVER

Remarks:

*RFI test*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = MAND      8164.13.00 06/09  
04/26/2016      14:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Gas	0.079	14:29
03 Room Air	0.000	14:30
04 Std. Gas	0.079	14:30
05 Room Air	0.000	14:31
06 Std. Gas	0.078	14:31
07 Room Air	0.000	14:32

Lot No = 29614080A2

Cyl No = 32

Exp Date = 12/05/2016

County = ~~08~~ 30  
DS

Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Calibration check 0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = MAND      8164.13.00 06/09  
04/26/2016      14:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:33
02 Std. Gas	0.080	14:33
03 Room Air	0.000	14:34
04 Std. Gas	0.079	14:34
05 Room Air	0.000	14:35
06 Std. Gas	0.078	14:35
07 Room Air	0.000	14:36

Lot No = 29614080A2  
Cyl No = 32  
Exp Date = 12/05/2016  
County = ~~08~~ 30      Oper No. = 777777  
          05

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:  
Calibration check 0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003067  
Location = MAND                8164.13.00 06/09  
04/26/2016                        14:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:37
02 Std. Gas	0.079	14:37
03 Room Air	0.000	14:37
04 Std. Gas	0.079	14:38
05 Room Air	0.000	14:38
06 Std. Gas	0.079	14:39
07 Room Air	0.000	14:39

Lot No = 29614080A2  
Cyl No = 32  
Exp Date = 12/05/2016  
County = ~~08~~ 30                    Oper No. = 777777  
          DS

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

Calibration check 0.080 AC