



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (1-2010)

Serial Number: <u>80-004938</u>	Instrument Location: <u>TIOGA</u>
Reason for Install/Repair:	
<input type="checkbox"/> Install After Lab Repair/Inspection <input checked="" type="checkbox"/> Other (Specify) <u>NEW INSTRUMENT</u>	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000. N/A
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

[Signature]
 Field Inspector Signature

4/27/10
 Date

Deb Kashner
 Reviewed By

10 May 2010
 Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = TIOG 8164.13.00 06/09
04/27/2010 20:35

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 21
Standard Expiration? 12/23/2011
Oper No? 020632

Flow Cal. Date: 01/25/2010
Slope 712
Intercept -604165

IR Calibration Date: 02/05/2010

	3um	9um
0th Coef(*100):	-32364	-29837
1st Coef(*100):	263210	133607
2nd Coef(*100):	406	576
H2O adj(mg/l*10k):	864	679

***** Printer Test End *****



Operator Signature
SHANNON P. DUISEN

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = TIOG 8164.13.00 06/09
04/27/2010 20:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:32
02 Std. Gas	0.078	20:33
03 Room Air	0.000	20:33
04 Std. Gas	0.078	20:33
05 Room Air	0.000	20:34
06 Std. Gas	0.077	20:34
07 Room Air	0.000	20:35

Lot No = 709352
Cyl No = 21
Exp Date = 12/23/2011
County = 53 Oper No. = 020632



Operator Signature
SHANNON P. DUISEN

Remarks:

Form 106-I8000

80-004938
R 47/48 PH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = TIOG 8164.13.00 06/09
04/27/2010 20:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	20:36
02 Room Air	0.000	20:37

*Invalid Test
Inhibited - RFI

Lot No = 709352
Cyl No = 21
Exp Date = 12/23/2011
County = 53 Oper No. = 020632



Operator Signature
SHANNON P. DUISEN

Remarks:

Form 106-I8000

80-004938
P. 78/48 192