



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number 80-003060	Instrument Location Mercer County Jail
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Changing of Gas Cylinder</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): Gas Cylinder Changed
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Shannan P. Lillard</i>	Date 9-9-11
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashur</i>	Date 27 Apr 2012

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = STAT 8164.13.00 06/09
09/09/2011 17:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 11611080A2
Standard Cyl #? 115
Standard Expiration? 06/01/2013
Oper No? 020904

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008
 3um 9um

0th Coef(*100): -5190 -7737
1st Coef(*100): 270580 138779
2nd Coef(*100): 2499 956
H2O adj(mg/l*10k): 377 188

***** Printer Test End *****



Operator Signature
SHANNON P. LIEBOLD

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = STAT 8164.13.00 06/09
09/09/2011 17:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:44
02 Std. Gas	0.079	17:45
03 Room Air	0.000	17:45
04 Std. Gas	0.079	17:45
05 Room Air	0.000	17:46
06 Std. Gas	0.079	17:46
07 Room Air	0.000	17:47

Lot No = 11611080A2
Cyl No = 115
Exp Date = 06/01/2013
County = 29 Oper No. = 020904



Operator Signature
SHANNON P. LIEBOLD

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = STAT 8164.13.00 06/09
09/09/2011 17:48

Test	AC	Time
01 Diagnostic	OK	17:52
02 Room Air	RFI*	17:52
03 Room Air	RFI*	17:52

*Invalid Test
Inhibited - RFI

Sub Name = TEST, RFI TEST
Sub DOB = 09/09/1980
Sub Sex = Male Weight = 320
Test = DUI Cit = 010101010
Dr. Lic. = ND/RFI123456789
Lot No = 11611080A2
Cyl No = 115
Expiration Date = 06/01/2013
County = 29 Oper No. = 020904

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
SHANNON P. LIEBOLD

Remarks: