

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004188

Location: BOWM

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $< 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3846 Lot No. FCS2 Rep AC 0.10 AC + 0.05% acetone ✓
- H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 31 Exp. Date 08/01/2014

Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 861 psi Reg 850 psi

K. Remarks/Maintenance record: Annual inspection, replaced simulator  
return o-ring

25 Apr 2014  
Date

28 APR 14  
Date

Deb Kashur  
Field Inspector's Signature

Charles E. Ed  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-004188  
Location = BOWM               8164.13.00 06/09  
04/25/2014                      11:10

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?           No  
Disable On Memfull?       Yes  
# of Print Copies?       1  
Select Std (D/W/I)?       Dry  
Standard Value?           0.080  
Standard Lot #?           16512080A2  
Standard Cyl #?           31  
Standard Expiration?       08/01/2014  
Oper No?                   777777

Flow Cal. Date:           07/05/2011  
Slope                      668  
Intercept                 -597204

IR Calibration Date:       04/05/2012  
                            3um                   9um

	3um	9um
0th Coef(*100):	-15090	-16406
1st Coef(*100):	263383	126966
2nd Coef(*100):	6533	2371
H2O adj(mg/l*10k):	801	809

\*\*\*\*\* Printer Test End \*\*\*\*\*

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = BOWM      8164.13.00 06/09  
04/25/2014      11:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:26
02 Std. Sol.	0.021	11:26
03 Room Air	0.000	11:27
04 Std. Sol.	0.022	11:28
05 Room Air	0.000	11:28
06 Std. Sol.	0.021	11:29
07 Room Air	0.000	11:29

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 06

Oper No. = 777777

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = BOWM      8164.13.00 06/09  
04/25/2014      11:18

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:19
02 Std. Sol.	0.296	11:19
03 Room Air	0.000	11:20
04 Std. Sol.	0.294	11:21
05 Room Air	0.000	11:21
06 Std. Sol.	0.293	11:22
07 Room Air	0.000	11:22

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 06

Oper No. = 777777

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

High AC 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = BOWM      8164.13.00 06/09  
04/25/2014      11:30

Test	AC	Time
01 Room Air	0.000	11:31
02 *Subject Test	INT*	11:31
03 Room Air	0.000	11:32

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 31  
Expiration Date = 08/01/2014  
County = 06      Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Interferent check*

*0.10AE +  
0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004188  
Location = BOWM            8164.13.00 06/09  
04/25/2014                            11:32

Test	AC	Time
01 Diagnostic	OK	11:33
02 Room Air	RFI*	11:33
03 Room Air	0.000	11:34

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                            Weight = 150  
Test = DUI    Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 31  
Expiration Date = 08/01/2014  
County = 06    Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*RFI check*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004188  
Location = BOWM                    8164.13.00 06/09  
04/25/2014                            10:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:56
02 Std. Gas	0.079	10:57
03 Room Air	0.000	10:57
04 Std. Gas	0.079	10:57
05 Room Air	0.000	10:58
06 Std. Gas	0.079	10:58
07 Room Air	0.000	10:59

Lot No = 16512080A2  
Cyl No = 31  
Exp Date = 08/01/2014  
County = 06                            Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = BOWM      8164.13.00 06/09  
04/25/2014      11:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:01
02 Std. Gas	0.080	11:02
03 Room Air	0.000	11:02
04 Std. Gas	0.079	11:02
05 Room Air	0.000	11:03
06 Std. Gas	0.080	11:03
07 Room Air	0.000	11:04

Lot No = 16512080A2  
Cyl No = 31  
Exp Date = 08/01/2014  
County = 06      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080AC



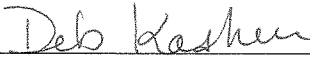
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004188  
Location = BOWM            8164.13.00 06/09  
04/25/2014                            11:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:07
02 Std. Gas	0.079	11:07
03 Room Air	0.000	11:07
04 Std. Gas	0.080	11:08
05 Room Air	0.000	11:08
06 Std. Gas	0.080	11:09
07 Room Air	0.000	11:09

Lot No = 16512080A2  
Cyl No = 31  
Exp Date = 08/01/2014  
County = 06                            Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check    0.080AC  
Form 106-I8000