

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004200

Location: MOTT

- |  | Check When Done |
|--|-----------------|
| A. Is the breath tube heated?  | <u>✓</u>        |
| B. Is the diagnostic check complete?   | <u>✓</u>        |
| C. Is the time, date, and year correct (re-set if necessary)?  | <u>✓</u>        |
| D. Print test. (Attach test record.)   | <u>✓</u>        |
| E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR5115</u> Lot No. <u>12111</u> Rep AC <u>0.020</u>  | <u>✓</u>        |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR5143</u> Lot No. <u>13680</u> Rep AC <u>0.306</u>  | <u>✓</u>        |
| G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.<br>(Attach test record.)<br>Sim SN <u>DR3846</u> Lot No. <u>FCS2</u> Rep AC <u>0.10 AC + 0.05% acetone</u> | <u>✓</u>        |
| H. RFI check. Run CMS mode. Key radio on first room air.<br>(Attach test record.)  | <u>✓</u>        |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.)  |                 |

Lot No. 16512680A2 Cyl No. 28 Exp. Date 08/01/2014

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 1019 psi Reg 1000 psi

K. Remarks/Maintenance record: Annual inspection, recalibrated

gas cylinder.

25 Apr 2014  
Date

28 APR 14  
Date

Deb Kashur  
Field Inspector's Signature

Charles E. Edr  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = MOTT      8164.13.00 06/09  
04/25/2014      13:52

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890--|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890--|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 16512080A2  
Standard Cyl #? 28  
Standard Expiration? 08/01/2014  
Oper No? 777777

Flow Cal. Date: 12/10/2012  
Slope 637  
Intercept -382487

IR Calibration Date: 12/10/2012  
3um 9um

0th Coef(*100):	-14942	-22738
1st Coef(*100):	256196	136921
2nd Coef(*100):	3535	1396
H2O adj(mg/l*10k):	492	515

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = MOTT      8164.13.00 06/09  
04/25/2014      14:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Sol.	0.020	14:12
03 Room Air	0.000	14:12
04 Std. Sol.	0.021	14:13
05 Room Air	0.000	14:14
06 Std. Sol.	0.020	14:14
07 Room Air	0.000	14:15

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 21

Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Low AC      0.020AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = MOTT      8164.13.00 06/09  
04/25/2014      14:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Sol.	0.303	14:06
03 Room Air	0.000	14:06
04 Std. Sol.	0.303	14:07
05 Room Air	0.000	14:07
06 Std. Sol.	0.302	14:08
07 Room Air	0.000	14:09

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 21

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*High AC    0.300 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004200  
Location = MOTT                    8164.13.00 06/09  
04/25/2014                            14:19

Test	AC	Time
01 Room Air	0.000	14:19
02 *Subject Test	INT*	14:20
03 Room Air	0.000	14:21

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 28  
Expiration Date = 08/01/2014  
County = 21                            Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Interferent check 0.10 AC + 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004200  
Location = MOTT                    8164.13.00 06/09  
04/25/2014                            14:21

Test	AC	Time
01 Diagnostic	OK	14:22
02 Room Air	RFI*	14:22
03 Room Air	0.000	14:23

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 28  
Expiration Date = 08/01/2014  
County = 21                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB KASHUR

Remarks:

*RFI check*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = MOTT      8164.13.00 06/09  
04/25/2014      13:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:53
02 Std. Gas	0.081	13:54
03 Room Air	0.000	13:54
04 Std. Gas	0.080	13:55
05 Room Air	0.000	13:55
06 Std. Gas	0.080	13:56
07 Room Air	0.000	13:56

Lot No = 16512080A2  
Cyl No = 28  
Exp Date = 08/01/2014  
County = 21      Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = MOTT      8164.13.00 06/09  
04/25/2014      13:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Gas	0.081	13:57
03 Room Air	0.000	13:58
04 Std. Gas	0.081	13:58
05 Room Air	0.000	13:59
06 Std. Gas	0.080	13:59
07 Room Air	0.000	13:59

Lot No = 16512080A2  
Cyl No = 28  
Exp Date = 08/01/2014  
County = 21      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = MOTT      8164.13.00 06/09  
04/25/2014      14:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:00
02 Std. Gas	0.080	14:01
03 Room Air	0.000	14:01
04 Std. Gas	0.081	14:02
05 Room Air	0.000	14:02
06 Std. Gas	0.080	14:02
07 Room Air	0.000	14:03

Lot No = 16512080A2  
Cyl No = 28  
Exp Date = 08/01/2014  
County = 21      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*