

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003061

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3066 Lot No. 15160 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3057 Lot No. 16102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 19615080A5 Cyl No. 003 Exp. Date 9/5/2017

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 378 psi Reg ~400 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED SIMULATOR RETURN O-RING, REPLACED INTERNAL PRINTER

4-27-2016
Date

Charles E. Eber
Field Inspector's Signature

28 Apr 16
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 15:12

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 3
Standard Expiration? 09/05/2017
Oper No? 666666

Flow Cal. Date: 03/04/2011
Slope 700
Intercept -623726

IR Calibration Date: 04/10/2015
 3um 9um

0th Coef(*100): -25722 -17879
1st Coef(*100): 266163 136703
2nd Coef(*100): 2302 1151
H2O adj(mg/l*10k): 412 294

***** Printer Test End *****

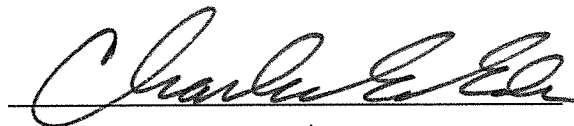

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 15:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:24
02 Std. Sol.	0.018	15:25
03 Room Air	0.000	15:25
04 Std. Sol.	0.018	15:26
05 Room Air	0.000	15:26
06 Std. Sol.	0.018	15:27
07 Room Air	0.000	15:28

08 Sim Temp = 34.0°C

Simul Ser No = MP3066
Std Sol No = 15160
County = 08 Oper No. = 666666

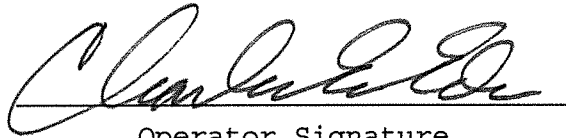

Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 15:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Sol.	0.399	15:42
03 Room Air	0.000	15:42
04 Std. Sol.	0.399	15:43
05 Room Air	0.000	15:44
06 Std. Sol.	0.398	15:44
07 Room Air	0.000	15:45


08 Sim Temp = 34.0°C

Simul Ser No = MP3057

Std Sol No = 16102

County = 08

Oper No. = 666666

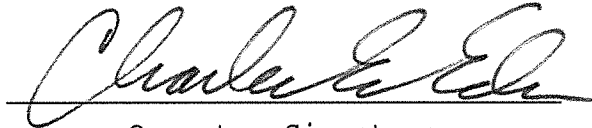

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 15:52

Test	AC	Time
01 Room Air	0.000	15:53
02 *Subject Test	INT*	15:53
03 Room Air	0.000	15:54

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 15:54

Test	AC	Time
01 Diagnostic	OK	15:55
02 Room Air	0.000	15:55
03 *Subject Test	RFI*	15:55
04 Room Air	0.000	15:56

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 15:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:57
02 Std. Gas	0.079	15:57
03 Room Air	0.000	15:58
04 Std. Gas	0.079	15:58
05 Room Air	0.000	15:59
06 Std. Gas	0.080	15:59
07 Room Air	0.000	15:59

Lot No = 19615080A5
Cyl No = 3
Exp Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 16:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:01
02 Std. Gas	0.079	16:01
03 Room Air	0.000	16:01
04 Std. Gas	0.080	16:02
05 Room Air	0.000	16:02
06 Std. Gas	0.080	16:03
07 Room Air	0.000	16:03

Lot No = 19615080A5

Cyl No = 3

Exp Date = 09/05/2017

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.13.00 06/09
04/27/2016 16:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:05
02 Std. Gas	0.079	16:05
03 Room Air	0.000	16:05
04 Std. Gas	0.080	16:06
05 Room Air	0.000	16:06
06 Std. Gas	0.079	16:07
07 Room Air	0.000	16:07

Lot No = 19615080A5
Cyl No = 3
Exp Date = 09/05/2017
County = 08 Oper No. = 666666

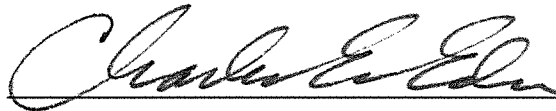

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000