



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (5-2011)

Serial Number <b>80-003071</b>	Instrument Location <b>BSPD</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A.  Print Test (Level 1, Function P).
  - B.  ACA Test (Level 1, Function C).
  - C.  RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): install
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Bob Kashner</i>	Date <i>29 Apr 2013</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Eln</i>	Date <i>30 APR 2013</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003071  
Location = BSPD                    8164.13.00 06/09  
04/29/2013                            10:59

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    16512080A2  
Standard Cyl #?                    36  
Standard Expiration?                08/01/2014  
Oper No?                            777777

Flow Cal. Date:                    07/09/2010  
Slope                                806  
Intercept                            -994737

IR Calibration Date:                04/06/2011  
    3um                                9um

-----  
0th Coef(\*100):                    -10362                                -4903  
1st Coef(\*100):                    272212                                136536  
2nd Coef(\*100):                    2666                                    1129  
H2O adj(mg/l\*10k):                537                                    411

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
DEB KASHUR

Remarks:



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = BSPD                    8164.13.00 06/09  
04/29/2013                            11:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:01
02 Std. Gas	0.080	11:01
03 Room Air	0.000	11:02
04 Std. Gas	0.081	11:02
05 Room Air	0.000	11:03
06 Std. Gas	0.081	11:03
07 Room Air	0.000	11:03

Lot No = 16512080A2

Cyl No = 36

Exp Date = 08/01/2014

County = 08

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check (ACA) 0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-003071  
Location = BSPD                              8164.13.00 06/09  
04/29/2013    11:04

Test	AC	Time
01 Diagnostic	OK	11:04
02 Room Air	RFI*	11:05
03 Room Air	0.000	11:05

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI    Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 36  
Expiration Date = 08/01/2014  
County = 08                              Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB KASHUR

Remarks:

*RFI check*

Form 106-I8000