

INTOXILYZER 8000 INSPECTION

Serial No.: 80-003065

Location: Bottineau Co SO

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>10290</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>1110</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR 3846</u> Lot No. <u>—</u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>05911080A1</u>	Cyl No. <u>19</u>	Exp. Date <u>02/01/2013</u>
Test 1 <u>0.080</u>	Test 4 <u>0.078</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.078</u>	Test 8 <u>0.078</u>
Test 3 <u>0.077</u>	Test 6 <u>0.077</u>	Test 9 <u>0.079</u>
Average <u>0.078</u>		

L. Tank pressure, Level 3 Func D Sub G P: 835 psi Reg 830 psi

M. Remarks/Maintenance record: Annual inspection

30 Apr 2012
Date

Deb Kasher
Field Inspector's Signature

01 MAY 12
Date

Charles E. Ehr
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = BOTT 8164.13.00 06/09
04/30/2012 12:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:38
02 Std. Sol.	0.019	12:39
03 Room Air	0.000	12:40
04 Std. Sol.	0.019	12:40
05 Room Air	0.000	12:41
06 Std. Sol.	0.020	12:42
07 Room Air	0.000	12:42

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 05

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks: Low AC 0.020 AC

~~Calibration check 0.080 AC DK~~

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = BOTT 8164.13.00 06/09
04/30/2012 12:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Sol.	0.291	12:45
03 Room Air	0.000	12:45
04 Std. Sol.	0.292	12:46
05 Room Air	0.000	12:47
06 Std. Sol.	0.293	12:47
07 Room Air	0.000	12:48

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 05

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = BOTT 8164.13.00 06/09
04/30/2012 12:50

Test	AC	Time
01 Room Air	0.000	12:51
02 *Subject Test	INT*	12:51
03 Room Air	0.000	12:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 05911080A1
Cyl No = 19
Expiration Date = 02/01/2013
County = 05 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = BOTT 8164.13.00 06/09
04/30/2012 12:53

Test	AC	Time
01 Diagnostic	OK	12:54
02 Room Air	RFI*	12:54
03 Room Air	0.000	12:55

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 05911080A1
Cyl No = 19
Expiration Date = 02/01/2013
County = 05 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = BOTT 8164.13.00 06/09
04/30/2012 12:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:20
02 Std. Gas	0.080	12:20
03 Room Air	0.000	12:21
04 Std. Gas	0.079	12:21
05 Room Air	0.000	12:22
06 Std. Gas	0.077	12:22
07 Room Air	0.000	12:22

Lot No = 05911080A1
Cyl No = 19
Exp Date = 02/01/2013
County = 05 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = BOTT 8164.13.00 06/09
04/30/2012 12:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:24
02 Std. Gas	0.078	12:24
03 Room Air	0.000	12:25
04 Std. Gas	0.078	12:25
05 Room Air	0.000	12:26
06 Std. Gas	0.077	12:26
07 Room Air	0.000	12:26

Lot No = 05911080A1
Cyl No = 19
Exp Date = 02/01/2013
County = 05 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

