



ETHANOL GAS STANDARD CYLINDER REPORT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>David Stramberg</i>		
Location <i>Grand Forks County Jail</i>	Intoxilyzer® Serial Number <i>80-003065</i>	
Gas Lot Number <i>14214080A6</i>	Gas Cylinder Number <i>52</i>	Gas Expiration Date <i>7/05/16</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0. 081 AC
 B. 0. 080 AC
 C. 0. 080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>David Stramberg</i>	Date <i>5/1/2015</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>04 May 15</i>

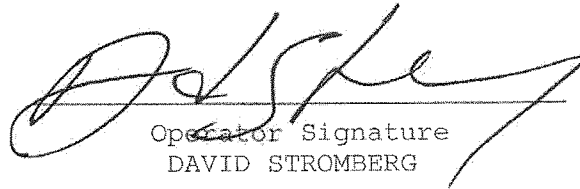
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
05/01/2015 03:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	03:58
02 Std. Gas	0.081	03:58
03 Room Air	0.000	03:59
04 Std. Gas	0.080	03:59
05 Room Air	0.000	04:00
06 Std. Gas	0.080	04:00
07 Room Air	0.000	04:01

Lot No = 14214080A6
Cyl No = 52
Exp Date = 07/05/2016
County = 18 Oper No. = 020836


Operator Signature
DAVID STROMBERG

Remarks:

*Gas cylinder change at Grand Forks
County jail*

Form 106-I8000