

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005955

Location: DUNN (DUNN Co. S.O.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5143 Lot No. 201501 B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3455 Lot No. 14270 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5132 Lot No. ICS #2 Solution 0.110AC + 0.05% ACE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 3391308044 Cyl No. 040 Exp. Date 1/1/2016

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 850 psi Reg 850 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5.1.15
Date

01 May 2015
Date

Charles E. Edler
Field Inspector's Signature

Deb Kashner
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:20

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 33913080A4
Standard Cyl #? 40
Standard Expiration? 01/01/2016
Oper No? 666666

Flow Cal. Date: 06/21/2013
Slope 757
Intercept -837385

IR Calibration Date: 06/21/2013
 3um 9um

0th Coef(*100): -19897 -22878
1st Coef(*100): 268381 131979
2nd Coef(*100): 5157 1473
H2O adj(mg/l*10k): 717 600

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:39
02 Std. Sol.	0.018	11:40
03 Room Air	0.000	11:40
04 Std. Sol.	0.019	11:41
05 Room Air	0.000	11:41
06 Std. Sol.	0.018	11:42
07 Room Air	0.000	11:43

08 Sim Temp = 34.0°C

Simul Ser No = DR5143
Std Sol No = 201501B
County = 13 Oper No. = 666666

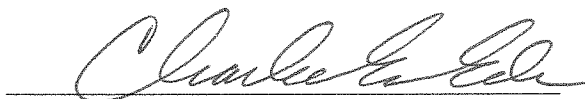

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:49
02 Std. Sol.	0.300	11:50
03 Room Air	0.000	11:51
04 Std. Sol.	0.300	11:51
05 Room Air	0.000	11:52
06 Std. Sol.	0.301	11:53
07 Room Air	0.000	11:53


08 Sim Temp = 34.0°C

Simul Ser No = DR3455

Std Sol No = 14270

County = 13

Oper No. = 666666

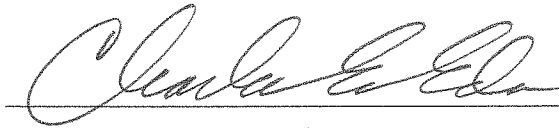

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:30

Test	AC	Time
01 Room Air	0.000	11:31
02 *Subject Test	INT*	11:31
03 Room Air	0.000	11:32

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 33913080A4
Cyl No = 40
Expiration Date = 01/01/2016
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:23

Test	AC	Time
01 Diagnostic	OK	11:24
02 Room Air	0.000	11:24
03 *Subject Test	RFI*	11:24
04 Room Air	0.000	11:25

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 33913080A4
Cyl No = 40
Expiration Date = 01/01/2016
County = 13 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

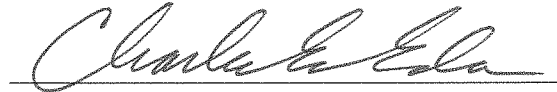

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:26
02 Std. Gas	0.082	11:27
03 Room Air	0.000	11:27
04 Std. Gas	0.082	11:27
05 Room Air	0.000	11:28
06 Std. Gas	0.081	11:28
07 Room Air	0.000	11:29

Lot No = 33913080A4
Cyl No = 40
Exp Date = 01/01/2016
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:34
02 Std. Gas	0.081	11:34
03 Room Air	0.000	11:35
04 Std. Gas	0.081	11:35
05 Room Air	0.000	11:36
06 Std. Gas	0.082	11:36
07 Room Air	0.000	11:37

Lot No = 33913080A4
Cyl No = 40
Exp Date = 01/01/2016
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/01/2015 11:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:54
02 Std. Gas	0.082	11:55
03 Room Air	0.000	11:55
04 Std. Gas	0.081	11:56
05 Room Air	0.000	11:56
06 Std. Gas	0.081	11:56
07 Room Air	0.000	11:57


Lot No = 33913080A4

Cyl No = 40

Exp Date = 01/01/2016

County = 13

Oper No. = 666666

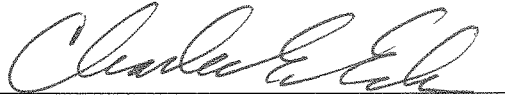

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000