



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

| | |
|---|---|
| Serial Number 80-005951 | Instrument Location Westford City, McKeown Co Sheriff |
| Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____ | |

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|---|--------------------------|
| Field Inspector Signature <i>[Signature]</i> | Date 5/1/15 ✓ |
| Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i> | Date 14 May 15 |

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. [Signature]* Certified Date: **21 Nov. 16**


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/01/2015 19:05

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 19:05 |
| 02 Std. Gas | 0.081 | 19:06 |
| 03 Room Air | 0.000 | 19:06 |
| 04 Std. Gas | 0.080 | 19:06 |
| 05 Room Air | 0.000 | 19:07 |
| 06 Std. Gas | 0.078 | 19:07 |
| 07 Room Air | 0.000 | 19:08 |

Lot No = 13314080A2
Cyl No = 050
Exp Date = 06/05/2016
County = 27 Oper No. = 030924

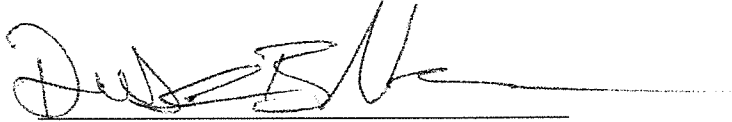

Operator Signature
DUSTIN NEWMAN

Remarks:

ACA Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/01/2015 19:08

| Test | AC | Time |
|---------------|------|-------|
| 01 Diagnostic | OK | 19:10 |
| 02 Room Air | RFI* | 19:10 |
| 03 Room Air | RFI* | 19:10 |

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 03/25/1980
Sub Sex = Male Weight = 225
Test = DUI Cit = 1234567
Dr. Lic. = ND/0123456789
Lot No = 13314080A2
Cyl No = 050
Expiration Date = 06/05/2016
County = 27 Oper No. = 030924

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

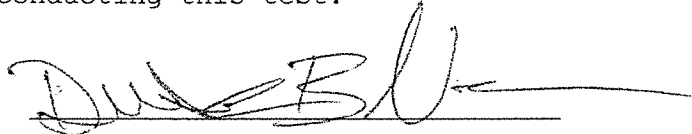

Operator Signature
DUSTIN NEWMAN

Remarks: RFI TEST