

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004184

Location: Grant Co SO

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>10290</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>1110</u> Rep AC <u>0.360</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>23411080A1</u>	Cyl No. <u>38</u>	Exp. Date <u>10/01/2013</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 990 psi Reg 1000 psi

M. Remarks/Maintenance record: Annual inspection

02 May 2012
Date

Deb Kashur
Field Inspector's Signature

02 MAY 2012
Date

Charles E. E...
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:09

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-==|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-==|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 38
Standard Expiration? 10/01/2013
Oper No? 777777

Flow Cal. Date: 07/30/2010
Slope 804
Intercept -925008

IR Calibration Date: 07/30/2010
 3um 9um

0th Coef(*100): -19612 -18975
1st Coef(*100): 270159 137378
2nd Coef(*100): 1174 963
H2O adj(mg/l*10k): 427 372

***** Printer Test End *****

Debo Kashur

Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:23
02 Std. Sol.	0.020	10:24
03 Room Air	0.000	10:24
04 Std. Sol.	0.021	10:25
05 Room Air	0.000	10:26
06 Std. Sol.	0.020	10:26
07 Room Air	0.000	10:27

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 19

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:28
02 Std. Sol.	0.303	10:29
03 Room Air	0.000	10:30
04 Std. Sol.	0.302	10:30
05 Room Air	0.000	10:31
06 Std. Sol.	0.301	10:32
07 Room Air	0.000	10:32

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 19

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:34

Test	AC	Time
01 Room Air	0.000	10:34
02 *Subject Test	INT*	10:34
03 Room Air	0.000	10:35

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 38
Expiration Date = 10/01/2013
County = 19 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:39

Test	AC	Time
01 Diagnostic	OK	10:40
02 Room Air	RF1*	10:40
03 Room Air	0.000	10:40

*Invalid Test
Inhibited - RF1

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/D15921456
Lot No = 23411080A1
Cyl No = 38
Expiration Date = 10/01/2013
County = 19 Oper No. = 777777

I followed the Approved Method
and the instructions displayed
by the Intoxilyzer in conducting
this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RF1 test

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:09
02 Std. Gas	0.080	10:10
03 Room Air	0.000	10:10
04 Std. Gas	0.080	10:11
05 Room Air	0.000	10:11
06 Std. Gas	0.080	10:11
07 Room Air	0.000	10:12

Lot No = 23411080A1
Cyl No = 38
Exp Date = 10/01/2013
County = 19 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 Ac

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:13
02 Std. Gas	0.080	10:13
03 Room Air	0.000	10:14
04 Std. Gas	0.081	10:14
05 Room Air	0.000	10:15
06 Std. Gas	0.080	10:15
07 Room Air	0.000	10:16

Lot No = 23411080A1
Cyl No = 38
Exp Date = 10/01/2013
County = 19 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = CARS 8164.13.00 06/09
05/02/2012 10:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:18
02 Std. Gas	0.080	10:18
03 Room Air	0.000	10:19
04 Std. Gas	0.080	10:19
05 Room Air	0.000	10:19
06 Std. Gas	0.080	10:20
07 Room Air	0.000	10:20

Lot No = 23411080A1
Cyl No = 38
Exp Date = 10/01/2013
County = 19 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC