



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <b>80-003060</b>	Instrument Location <b>CARR - Foster Co SO</b> <i>JK</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <b>INSTALL DURING ANNUAL INSPECTION</b>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): **NA - INSTALL DURING ANNUAL INSP.**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Eden</i>	Date <b>03 MAY 13</b>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <b>06 May 13</b>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003060  
Location = CARR                8164.13.00 06/09  
05/03/2013                      13:47

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            11  
Standard Expiration?       08/01/2014  
Oper No?                    666666

Flow Cal. Date:            04/08/2008  
Slope                      702  
Intercept                  -647610

IR Calibration Date:        06/04/2008

	3um	9um
0th Coef(*100):	-5190	-7737
1st Coef(*100):	270580	138779
2nd Coef(*100):	2499	956
H2O adj(mg/l*10k):	377	188

\*\*\*\*\* Printer Test End \*\*\*\*\*

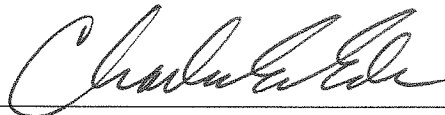

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003060  
Location = CARR                8164.13.00 06/09  
05/03/2013                        13:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:52
02 Std. Gas	0.080	13:53
03 Room Air	0.000	13:53
04 Std. Gas	0.080	13:53
05 Room Air	0.000	13:54
06 Std. Gas	0.080	13:54
07 Room Air	0.000	13:55

Lot No = 16512080A2  
Cyl No = 11  
Exp Date = 08/01/2014  
County = 16                            Oper No. = 666666

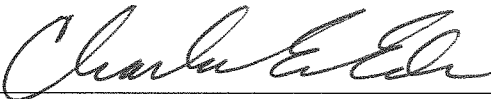

Operator Signature  
CHARLES EDER

Remarks:    *ACA TEST*  
                  *0.080 AC (E.G.S.)*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003060  
Location = CARR                    8164.13.00 06/09  
05/03/2013                            13:57

Test	AC	Time
01 Diagnostic	OK	13:58
02 Room Air	RFI*	13:58
03 Room Air	0.000	13:58

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = QWERTYUIOPASDFG  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 11  
Expiration Date = 08/01/2014  
County = 16                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

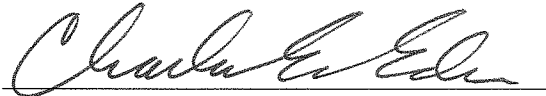

Operator Signature  
CHARLES EDER

Remarks: **RFI TEST**