



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number 80-003060	Instrument Location CARR - Foster Co SO BK
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) INSTALL DURING ANNUAL INSPECTION	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): NA - INSTALL DURING ANNUAL INSP.
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Edin</i>	Date 03 MAY 13
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>06 May 13</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edin* Certified Date: **04 Nov. 16**

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = CARR 8164.13.00 06/09
05/03/2013 13:47

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 11
Standard Expiration? 08/01/2014
Oper No? 666666

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008
 3um 9um

0th Coef(*100):	-5190	-7737
1st Coef(*100):	270580	138779
2nd Coef(*100):	2499	956
H2O adj(mg/l*10k):	377	188

***** Printer Test End *****

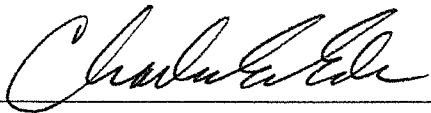

Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = CARR 8164.13.00 06/09
05/03/2013 13:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:52
02 Std. Gas	0.080	13:53
03 Room Air	0.000	13:53
04 Std. Gas	0.080	13:53
05 Room Air	0.000	13:54
06 Std. Gas	0.080	13:54
07 Room Air	0.000	13:55

Lot No = 16512080A2
Cyl No = 11
Exp Date = 08/01/2014
County = 16 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ACA TEST
0.080 AC (E.G.S.)

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = CARR 8164.13.00 06/09
05/03/2013 13:57

Test	AC	Time
01 Diagnostic	OK	13:58
02 Room Air	RFI*	13:58
03 Room Air	0.000	13:58

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = QWERTYUIOPASDFG
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 11
Expiration Date = 08/01/2014
County = 16 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: **RFI TEST**