



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number 80-004186	Instrument Location STEE
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) INSTALL DURING ANNUAL INSPECTION ON SITE	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000. **(SHARE W/FAX)**
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): **NA - INSTALL**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature Charles E. Edr	Date 03 MAY 13
Reviewed By (Crime Laboratory Use Only) Deb Kashner	Date 04 May 13

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = STEE 8164.13.00 06/09
05/03/2013 09:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 14
Standard Expiration? 08/01/2014
Oper No? 666666

Flow Cal. Date: 06/03/2009
Slope 703
Intercept -668211

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100): -12098 -31908
1st Coef(*100): 257966 136312
2nd Coef(*100): 2942 1009
H2O adj(mg/l*10k): 434 609

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = STEE 8164.13.00 06/09
05/03/2013 09:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:34
02 Std. Gas	0.081	09:35
03 Room Air	0.000	09:35
04 Std. Gas	0.080	09:36
05 Room Air	0.000	09:36
06 Std. Gas	0.081	09:36
07 Room Air	0.000	09:37

Lot No = 16512080A2
Cyl No = 14
Exp Date = 08/01/2014
County = 22 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *ACA TEST*
 0.080 AC (E.G.S.)

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = STEE 8164.13.00 06/09
05/03/2013 09:38

Test	AC	Time
01 Diagnostic	OK	09:39
02 Room Air	RFI*	09:39
03 Room Air	0.000	09:40

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = QWERTYUIOPASDFG
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 14
Expiration Date = 08/01/2014
County = 22 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI TEST