

INTOXILYZER 8000 INSPECTION

Serial No.: 80-003066

Location: DKPD

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5115</u> Lot No. <u> </u> Rep AC <u>0.298</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5186</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>659358</u>	Cyl No. <u>23</u>	Exp. Date <u>09/03/2011</u>
Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

L. Tank pressure, Level 3 Func D Sub G P: 373 psi Reg 370 psi

M. Remarks/Maintenance record: Annual inspection

<u>04 May 2010</u> Date	<u>Deb Kashur</u> Field Inspector's Signature
<u>15 May 2010</u> Date	<u>Mary Pearson</u> Reviewed by

80-003066
P. 86 of 94
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 07:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 23
Standard Expiration? 09/03/2011
Oper No? 777777

Flow Cal. Date: 04/08/2008
Slope 696
Intercept -643215

IR Calibration Date: 06/04/2008

	3um	9um
0th Coef(*100):	-11861	-9861
1st Coef(*100):	275638	138672
2nd Coef(*100):	165	672
H2O adj(mg/l*10k):	357	250

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-18000

80-003066
P. 87 of 94
SAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 08:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:04
02 Std. Sol.	0.017	08:05
03 Room Air	0.000	08:05
04 Std. Sol.	0.018	08:06
05 Room Air	0.000	08:06
06 Std. Sol.	0.018	08:07
07 Room Air	0.000	08:08

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 45

Oper No. = 77777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.021 AC

Form 106-I8000

80-003066
P. 88 of 94
km

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 08:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:15
02 Std. Sol.	0.286	08:16
03 Room Air	0.000	08:17
04 Std. Sol.	0.287	08:18
05 Room Air	0.000	08:18
06 Std. Sol.	0.289	08:19
07 Room Air	0.000	08:19

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 45

Oper No. = 77777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity check ^{test} OK

Form 106-18000

80-003066
P. 89 of 94
EK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 08:00

Test	AC	Time
01 Room Air	0.000	08:01
02 *Subject Test	INT*	08:01
03 Room Air	0.000	08:02

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 23
Expiration Date = 09/03/2011
County = 45 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check

Form 106-I8000

80-003066
P.90 of 94
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 07:58

Test	AC	Time
01 Diagnostic	OK	07:59
02 Room Air	RFI*	07:59
03 Room Air	0.000	08:00

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 23
Expiration Date = 09/03/2011
County = 45 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

80-003066
P. 91 of 74
JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 07:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:44
02 Std. Gas	0.081	07:44
03 Room Air	0.000	07:45
04 Std. Gas	0.082	07:45
05 Room Air	0.000	07:46
06 Std. Gas	0.082	07:46
07 Room Air	0.000	07:47

Lot No = 659358
Cyl No = 23
Exp Date = 09/03/2011
County = 45 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-18000

80-003066
P. 92 of 94
JKM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 07:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:48
02 Std. Gas	0.082	07:48
03 Room Air	0.000	07:49
04 Std. Gas	0.082	07:49
05 Room Air	0.000	07:50
06 Std. Gas	0.081	07:50
07 Room Air	0.000	07:50

Lot No = 659358
Cyl No = 23
Exp Date = 09/03/2011
County = 45 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003066
P. 93 of 94
JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = DKPD 8164.13.00 06/09
05/04/2010 07:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:52
02 Std. Gas	0.081	07:52
03 Room Air	0.000	07:53
04 Std. Gas	0.081	07:53
05 Room Air	0.000	07:54
06 Std. Gas	0.082	07:54
07 Room Air	0.000	07:55

Lot No = 659358
Cyl No = 23
Exp Date = 09/03/2011
County = 45 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.081 AC

Form 106-I8000

80-003066
P. 94 of 94
JAM