



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (1-2010)

Serial Number: <u>80-003071</u>	Instrument Location: <u>Coopers town</u>
Reason for Install/Repair:	
<input checked="" type="checkbox"/> Install After Lab Repair/Inspection <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

[Signature]
Field Inspector Signature

05-04-11
Date

Deb Kashur
Reviewed By

06 May 2011
Date

80-003071
P.133 of 136
[Signature]

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = COOP 8164.13.00 06/09
05/04/2011 14:13

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05911080A1
Standard Cyl #? 5
Standard Expiration? 02/01/2013
Oper No? 011017

Flow Cal. Date: 07/09/2010
Slope 806
Intercept -994737

IR Calibration Date: 04/06/2011

	3um	9um
0th Coef(*100):	-10362	-4903
1st Coef(*100):	272212	136536
2nd Coef(*100):	2666	1129
H2O adj(mg/l*10k):	537	411

***** Printer Test End *****

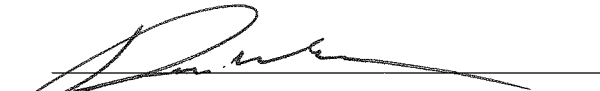

Operator Signature
JUSTIN A. WALLERICH

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = COOP 8164.13.00 06/09
05/04/2011 14:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:19
02 Std. Gas	0.079	14:19
03 Room Air	0.000	14:20
04 Std. Gas	0.079	14:20
05 Room Air	0.000	14:21
06 Std. Gas	0.080	14:21
07 Room Air	0.000	14:21

Lot No = 05911080A1
Cyl No = 5
Exp Date = 02/01/2013
County = 20 Oper No. = 011017



Operator Signature
JUSTIN A. WALLERICH

Remarks:

Form 106-I8000

80-003071
P.135 of 136
JAW

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = COOP 8164.13.00 06/09
05/04/2011 14:25

Test	AC	Time
01 Diagnostic	OK	14:26
02 Room Air	RFI*	14:26
03 Room Air	0.000	14:27

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST A
Sub DOB = 12/31/1972
Sub Sex = Male Weight = 200
Test = DUI Cit = 445544
Dr. Lic. = ND/NA
Lot No = 05911080A1
Cyl No = 5
Expiration Date = 02/01/2013
County = 20 Oper No. = 011017

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
JUSTIN A. WALLERICH

Remarks:

Form 106-I8000

80-003071
p. 136 of 136
JAW