

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004203

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u>—</u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3847</u> Lot No. <u>—</u> Rep AC <u>0.294</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3452</u> Lot No. <u>—</u> Rep AC <u>0.10AC+Acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>15410080A1</u>	Cyl No. <u>21</u>	Exp. Date <u>7-1-12</u>
Test 1 <u>0.079</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.079</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.082</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 305 psi Reg 300 psi

M. Remarks/Maintenance record: Annual Inspection

5-4-11
Date

[Signature]
Field Inspector's Signature

05 May 2011
Date

Deb Kashur
Reviewed by

80-004203
P.106 of 114
LKM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = ~~COOR~~ *10XL* 8164.13.00 06/09
05/04/2011 09:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 21
Standard Expiration? 07/01/2012
Oper No? 555555

Flow Cal. Date: 01/28/2010
Slope 719
Intercept -711903

IR Calibration Date: 01/28/2010
 3um 9um

0th Coef(*100): -17469 -28710
1st Coef(*100): 264216 135022
2nd Coef(*100): 2778 1081
H2O adj(mg/l*10k): 430 605

***** Printer Test End *****



Operator Signature
ASHLEY AYD

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 09:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:59
02 Std. Sol.	0.018	10:00
03 Room Air	0.000	10:01
04 Std. Sol.	0.019	10:01
05 Room Air	0.000	10:02
06 Std. Sol.	0.018	10:02
07 Room Air	0.000	10:03

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks: *Low AC 0.021AC*

Form 106-I8000

*80-004203
P. 108 of 114
FM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 10:05

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:05
02 Std. Sol.	0.286	10:06
03 Room Air	0.000	10:07
04 Std. Sol.	0.288	10:08
05 Room Air	0.000	10:08
06 Std. Sol.	0.289	10:09
07 Room Air	0.000	10:09


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

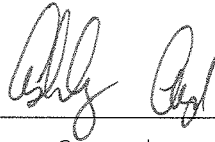

Operator Signature
ASHLEY AYD

Remarks: *Linearity test 0.294 AC*

Form 106-I8000

80-004203
0.109 of 114
PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 10:11

Test	AC	Time
01 Room Air	0.000	10:11
02 *Subject Test	INT*	10:12
03 Room Air	0.000	10:13

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 15410080A1
Cyl No = 21
Expiration Date = 07/01/2012
County = 08 Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks: *Interferent check 0.10 AC+Acetone*

Form 106-I8000

80-004203
P.110 of 114
JAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 10:13

Test	AC	Time
01 Diagnostic	OK	10:14
02 Room Air	RFI*	10:14
03 Room Air	0.000	10:15

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 15410080A1
Cyl No = 21
Expiration Date = 07/01/2012
County = 08 Oper No. = 555555

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ASHLEY AYD

Remarks: RFI

Form 106-I8000

80-004203
P. 111 of 114
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 09:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:55
02 Std. Gas	0.079	09:55
03 Room Air	0.000	09:56
04 Std. Gas	0.079	09:56
05 Room Air	0.000	09:57
06 Std. Gas	0.080	09:57
07 Room Air	0.000	09:57

Lot No = 15410080A1
Cyl No = 21
Exp Date = 07/01/2012
County = 08 Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks: Calibration Check 0.080 AC

Form 106-I8000

80-004203
0.112 of 114
KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 10:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:16
02 Std. Gas	0.082	10:16
03 Room Air	0.000	10:16
04 Std. Gas	0.081	10:17
05 Room Air	0.000	10:17
06 Std. Gas	0.082	10:18
07 Room Air	0.000	10:18

Lot No = 15410080A1
Cyl No = 21
Exp Date = 07/01/2012
County = 08 Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks:

Form 106-I8000

80-004203
P. 113 of 114
EM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = TOXL 8164.13.00 06/09
05/04/2011 13:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:32
02 Std. Gas	0.082	13:32
03 Room Air	0.000	13:33
04 Std. Gas	0.080	13:33
05 Room Air	0.000	13:34
06 Std. Gas	0.080	13:34
07 Room Air	0.000	13:34

Lot No = 15410080A1
Cyl No = 21
Exp Date = 07/01/2012
County = 08 Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks: Calibration check 0.080 AC

Form 106-I8000

80-004203
P.114 of 114
KRM