

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004943

Location: STAN (MOUNTAIN G.S.O.)

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) Sim SN <u>DR3455</u> Lot No. <u>10290</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) Sim SN <u>DR3847</u> Lot No. <u>11110</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) Sim SN <u>DR3377</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.) | |

Lot No. 23411080A1 Cyl No. 08 Exp. Date 10/01/2013

Test 1 0.077 Test 4 0.078 Test 7 0.079

Test 2 0.078 Test 5 0.078 Test 8 0.079

Test 3 0.079 Test 6 0.078 Test 9 0.079

Average 0.078

L. Tank pressure, Level 3 Func D Sub G P: 904 psi Reg 900 psi

M. Remarks/Maintenance record: ANNUAL INSPECTION

04 MAY 12
Date

Charles E. Ed
Field Inspector's Signature

07 May 2012
Date

Deb Kashur
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 11:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 08
Standard Expiration? 10/01/2013
Oper No? 666666

Flow Cal. Date: 01/18/2010
Slope 693
Intercept -665387

IR Calibration Date: 01/18/2010
 3um 9um

0th Coef(*100): -20351 -24446
1st Coef(*100): 248542 131200
2nd Coef(*100): 2180 1190
H2O adj(mg/l*10k): 594 450

***** Printer Test End *****

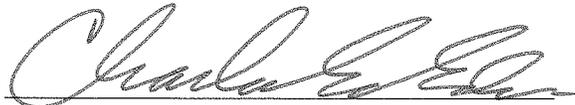

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 12:08

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 12:08 |
| 02 Std. Sol. | 0.016 | 12:09 |
| 03 Room Air | 0.000 | 12:10 |
| 04 Std. Sol. | 0.016 | 12:10 |
| 05 Room Air | 0.000 | 12:11 |
| 06 Std. Sol. | 0.016 | 12:12 |
| 07 Room Air | 0.000 | 12:12 |

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 31

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 12:19

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 12:20 |
| 02 Std. Sol. | 0.299 | 12:21 |
| 03 Room Air | 0.000 | 12:21 |
| 04 Std. Sol. | 0.297 | 12:22 |
| 05 Room Air | 0.000 | 12:22 |
| 06 Std. Sol. | 0.297 | 12:23 |
| 07 Room Air | 0.000 | 12:24 |

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 31

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

LINEARITY TEST
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 12:29

| Test | AC | Time |
|---------------|-------|-------|
| 01 Diagnostic | OK | 12:30 |
| 02 Room Air | RFI* | 12:30 |
| 03 Room Air | 0.000 | 12:30 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = MNBVCXZLKJHG
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 0000
County = 31 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 12:33

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:33 |
| 02 Std. Gas | 0.077 | 12:34 |
| 03 Room Air | 0.000 | 12:34 |
| 04 Std. Gas | 0.078 | 12:35 |
| 05 Room Air | 0.000 | 12:35 |
| 06 Std. Gas | 0.079 | 12:35 |
| 07 Room Air | 0.000 | 12:36 |

Lot No = 23411080A1
Cyl No = 8
Exp Date = 10/01/2013
County = 31 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 12:36

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:37 |
| 02 Std. Gas | 0.078 | 12:37 |
| 03 Room Air | 0.000 | 12:38 |
| 04 Std. Gas | 0.078 | 12:38 |
| 05 Room Air | 0.000 | 12:39 |
| 06 Std. Gas | 0.078 | 12:39 |
| 07 Room Air | 0.000 | 12:39 |

Lot No = 23411080A1
Cyl No = 8
Exp Date = 10/01/2013
County = 31 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = STAN 8164.13.00 06/09
05/04/2012 12:40

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:41 |
| 02 Std. Gas | 0.079 | 12:41 |
| 03 Room Air | 0.000 | 12:41 |
| 04 Std. Gas | 0.079 | 12:42 |
| 05 Room Air | 0.000 | 12:42 |
| 06 Std. Gas | 0.079 | 12:43 |
| 07 Room Air | 0.000 | 12:43 |

Lot No = 23411080A1
Cyl No = 8
Exp Date = 10/01/2013
County = 31 Oper No. = 666666

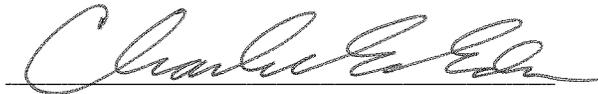

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC