



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number 80-003067	Instrument Location Morton Co Corr (Mandan PD) ^{PRIPR}
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): N/A install
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- N/A 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature Deb Kashur	Date 04 May 15 ✓
Reviewed By (Crime Laboratory Use Only) Charles E. Eder	Date 5.4.15

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: **Charles E. Eder** Certified Date: **30 Nov. 2016**

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = MAND 8164.13.00 06/09
05/04/2015 13:40

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 29614080A2
Standard Cyl #? 32
Standard Expiration? 12/05/2016
Oper No? 777777

Flow Cal. Date: 01/23/2013
Slope 664
Intercept -509770

IR Calibration Date: 04/30/2015
 3um 9um

0th Coef(*100): -19633 -12115
1st Coef(*100): 278224 137001
2nd Coef(*100): 2596 1415
H2O adj(mg/l*10k): 323 247

***** Printer Test End *****

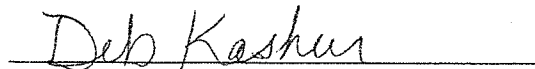

Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = MAND 8164.13.00 06/09
05/04/2015 13:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:46
02 Std. Gas	0.080	13:47
03 Room Air	0.000	13:47
04 Std. Gas	0.079	13:48
05 Room Air	0.000	13:48
06 Std. Gas	0.081	13:48
07 Room Air	0.000	13:49

Lot No = 29614080A2
Cyl No = 32
Exp Date = 12/05/2016
County = 30 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

ACA during install

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = MAND 8164.13.00 06/09
05/04/2015 13:49

Test	AC	Time
01 Diagnostic	OK	13:50
02 Room Air	RFI*	13:50
03 Room Air	0.000	13:51

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 32
Expiration Date = 12/05/2016
County = 30 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000