

INTOXILYZER 8000 INSPECTION

Serial No.: 80-003065

Location: Divide Co So - Crosby

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5115</u> Lot No. <u> </u> Rep AC <u>0.298</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5186</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>659358</u>	Cyl No. <u>6</u>	Exp. Date <u>09/03/2011</u>
Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

L. Tank pressure, Level 3 Func D Sub G P: 868 psi Reg 870 psi

M. Remarks/Maintenance record: Annual inspection

05 May 2010
Date

Deb Kashur
Field Inspector's Signature

15 May 2010
Date

Mary Pearson
Reviewed by

80-003065
p. 79 of 82
VFL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Sol.	0.021	14:24
03 Room Air	0.000	14:24
04 Std. Sol.	0.021	14:25
05 Room Air	0.000	14:25
06 Std. Sol.	0.021	14:26
07 Room Air	0.000	14:27

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 12

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.021 AC

Form 106-18000

80-003065
P. 76 of 82
JRM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:28
02 Std. Sol.	0.303	14:30
03 Room Air	0.000	14:30
04 Std. Sol.	0.304	14:31
05 Room Air	0.000	14:31
06 Std. Sol.	0.302	14:32
07 Room Air	0.000	14:33

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 12

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.298AC

Form 106-I8000

80-003065
p. 77 of 82
KH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:38

Test	AC	Time
01 Room Air	0.000	14:39
02 *Subject Test	INT*	14:39
03 Room Air	0.000	14:40

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 12 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

80-003065
P. 78 of 82
JKM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	14:04
02 Room Air	0.000	14:04

*Invalid Test
Inhibited - RFI

Lot No = 659358
Cyl No = 6
Exp Date = 09/03/2011
County = 12 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-18000

80-003065
P. 79 of 82
KEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:01
02 Std. Gas	0.082	14:01
03 Room Air	0.000	14:02
04 Std. Gas	0.082	14:02
05 Room Air	0.000	14:03
06 Std. Gas	0.081	14:03
07 Room Air	0.000	14:03

Lot No = 659358

Cyl No = 6

Exp Date = 09/03/2011

County = 12

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003065
P. 80 of 82
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:06
02 Std. Gas	0.082	14:06
03 Room Air	0.000	14:06
04 Std. Gas	0.081	14:07
05 Room Air	0.000	14:07
06 Std. Gas	0.081	14:08
07 Room Air	0.000	14:08

Lot No = 659358

Cyl No = 6

Exp Date = 09/03/2011

County = 12

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003065
P. 81 of 82
JKH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = CROS 8164.13.00 06/09
05/05/2010 14:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:10
02 Std. Gas	0.081	14:11
03 Room Air	0.000	14:11
04 Std. Gas	0.081	14:12
05 Room Air	0.000	14:12
06 Std. Gas	0.081	14:12
07 Room Air	0.000	14:13

Lot No = 659358

Cyl No = 6

Exp Date = 09/03/2011

County = 12

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003065
P. 82 of 82
SKH