

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005951

Location: WATF (McKENZIE COUNTY S.B.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3803 Lot No. 15160 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5190 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 13314080A2 Cyl No. 050 Exp. Date 06/05/2016

Test 1 <u>0.080</u>	Test 4 <u>0.078</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.078</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 865 psi Reg ~875 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/2/2016
Date

Charles E. Edr
Field Inspector's Signature

05 May 16
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 13:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A2
Standard Cyl #? 050
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 06/21/2013
Slope 693
Intercept -721277

IR Calibration Date: 04/24/2015
 3um 9um

	3um	9um
0th Coef(*100):	-27987	-24018
1st Coef(*100):	264819	135902
2nd Coef(*100):	2881	1395
H2O adj(mg/l*10k):	594	504

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: **PRINT TEST**

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-005951
Location = WATF	8164.13.00 06/09
05/02/2016	14:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Sol.	0.019	14:05
03 Room Air	0.000	14:06
04 Std. Sol.	0.019	14:07
05 Room Air	0.000	14:07
06 Std. Sol.	0.019	14:08
07 Room Air	0.000	14:08

08 Sim Temp = 34.0°C

Simul Ser No = DR3803
 Std Sol No = 15160
 County = 27 Oper No. = 666666



Operator Signature
 CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 14:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:09
02 Std. Sol.	0.305	14:10
03 Room Air	0.000	14:11
04 Std. Sol.	0.306	14:11
05 Room Air	0.000	14:12
06 Std. Sol.	0.306	14:13
07 Room Air	0.000	14:13

08 Sim Temp = 34.0°C

Simul Ser No = DR3847
Std Sol No = 16060
County = 27 Oper No. = 666666

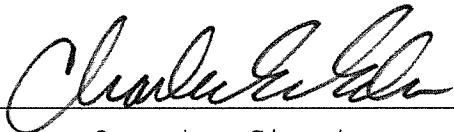

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 14:14

Test	AC	Time
01 Room Air	0.000	14:15
02 *Subject Test	INT*	14:16
03 Room Air	0.000	14:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A2
Cyl No = 050
Expiration Date = 06/05/2016
County = 27 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 14:17

Test	AC	Time
01 Diagnostic	OK	14:17
02 Room Air	0.000	14:18
03 *Subject Test	RFI*	14:18
04 Room Air	0.000	14:19

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A2
Cyl No = 050
Expiration Date = 06/05/2016
County = 27 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 14:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:19
02 Std. Gas	0.080	14:20
03 Room Air	0.000	14:20
04 Std. Gas	0.079	14:21
05 Room Air	0.000	14:21
06 Std. Gas	0.078	14:22
07 Room Air	0.000	14:22

Lot No = 13314080A2
Cyl No = 050
Exp Date = 06/05/2016
County = 27 Oper No. = 666666

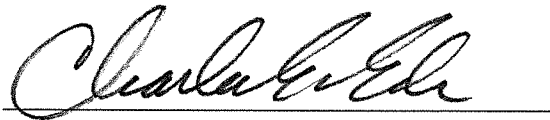

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 14:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Gas	0.078	14:23
03 Room Air	0.000	14:24
04 Std. Gas	0.079	14:24
05 Room Air	0.000	14:25
06 Std. Gas	0.079	14:25
07 Room Air	0.000	14:25

Lot No = 13314080A2
Cyl No = 050
Exp Date = 06/05/2016
County = 27 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: **CALIBRATION CHECK**
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = WATF 8164.13.00 06/09
05/02/2016 14:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:26
02 Std. Gas	0.079	14:26
03 Room Air	0.000	14:27
04 Std. Gas	0.079	14:27
05 Room Air	0.000	14:28
06 Std. Gas	0.079	14:28
07 Room Air	0.000	14:29

Lot No = 13314080A2
Cyl No = 050
Exp Date = 06/05/2016
County = 27 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC