

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005955

Location: DUNN

(DUNN COUNTY S.O.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3803 Lot No. 15160 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5190 Lot No. JCS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 19615080A5 Cyl No. 029 Exp. Date 9/5/2017

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.083</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.083</u>	Test 6 <u>0.082</u>	Test 9 <u>0.083</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 190 psi Reg 225 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/2/2016
Date

05 May 16
Date

[Signature]
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:39

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 29
Standard Expiration? 09/05/2017
Oper No? 666666

Flow Cal. Date: 06/21/2013
Slope 757
Intercept -837385

IR Calibration Date: 06/21/2013

	3um	9um
0th Coef(*100):	-19897	-22878
1st Coef(*100):	268381	131979
2nd Coef(*100):	5157	1473
H2O adj(mg/l*10k):	717	600

***** Printer Test End *****

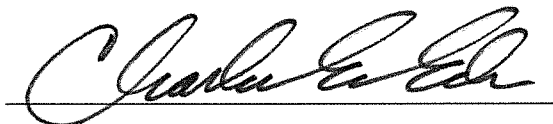

Operator Signature
CHARLES EDER

Remarks: **PRINT TEST**

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:53
02 Std. Sol.	0.020	09:54
03 Room Air	0.000	09:55
04 Std. Sol.	0.020	09:55
05 Room Air	0.000	09:56
06 Std. Sol.	0.019	09:57
07 Room Air	0.000	09:57

08 Sim Temp = 34.0°C

Simul Ser No = DR3803
Std Sol No = 15160
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:47
02 Std. Sol.	0.300	09:48
03 Room Air	0.000	09:48
04 Std. Sol.	0.304	09:49
05 Room Air	0.000	09:50
06 Std. Sol.	0.308	09:50
07 Room Air	0.000	09:51

08 Sim Temp = 34.0°C

Simul Ser No = DR3847
Std Sol No = 16060
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:42

Test	AC	Time
01 Room Air	0.000	09:43
02 *Subject Test	INT*	09:43
03 Room Air	0.000	09:44

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 29
Expiration Date = 09/05/2017
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:59

Test	AC	Time
01 Diagnostic	OK	10:00
02 Room Air	0.000	10:00
03 *Subject Test	RFI*	10:00
04 Room Air	0.000	10:01

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 029
Expiration Date = 09/05/2017
County = 13 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:27
02 Std. Gas	0.082	09:27
03 Room Air	0.000	09:28
04 Std. Gas	0.082	09:28
05 Room Air	0.000	09:29
06 Std. Gas	0.083	09:29
07 Room Air	0.000	09:30

Lot No = 19615080A5
Cyl No = 29
Exp Date = 09/05/2017
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 09:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:31
02 Std. Gas	0.082	09:31
03 Room Air	0.000	09:32
04 Std. Gas	0.082	09:32
05 Room Air	0.000	09:32
06 Std. Gas	0.082	09:33
07 Room Air	0.000	09:33

Lot No = 19615080A5
Cyl No = 29
Exp Date = 09/05/2017
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.13.00 06/09
05/02/2016 10:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:02
02 Std. Gas	0.083	10:02
03 Room Air	0.000	10:03
04 Std. Gas	0.082	10:03
05 Room Air	0.000	10:04
06 Std. Gas	0.083	10:04
07 Room Air	0.000	10:04

Lot No = 19615080A5
Cyl No = 029
Exp Date = 09/05/2017
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000