



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number 80-004200	Instrument Location HETTINGER COUNTY S.O. (MOTT)
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) INSTALL DURING ANNUAL INSPECTION	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000. **NA - SHARE W/FAX**
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C). ✓
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air). ✓
- 7. Repair and/or Maintenance Performed (if Any): **INSTALL DURING ANNUAL INSPECTION**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature CHARLES. E. EDER	Date 06 MAY 13
Reviewed By (Crime Laboratory Use Only) Deb Kashner	Date 07 May 13

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = MOTT 8164.13.00 06/09
05/06/2013 10:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 60
Standard Expiration? 10/01/2013
Oper No? 666666

Flow Cal. Date: 12/10/2012
Slope 637
Intercept -382487

IR Calibration Date: 12/10/2012

	3um	9um
0th Coef(*100):	-14942	-22738
1st Coef(*100):	256196	136921
2nd Coef(*100):	3535	1396
H2O adj(mg/l*10k):	492	515

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = MOTT 8164.13.00 06/09
05/06/2013 10:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:45
02 Std. Gas	0.080	10:46
03 Room Air	0.000	10:46
04 Std. Gas	0.080	10:46
05 Room Air	0.000	10:47
06 Std. Gas	0.080	10:47
07 Room Air	0.000	10:48

Lot No = 23411080A1
Cyl No = 60
Exp Date = 10/01/2013
County = 21 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ACA TEST
0.080 AC (E.G.S.)

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = MOTT 8164.13.00 06/09
05/06/2013 10:51

Test	AC	Time
01 Diagnostic	OK	10:52
02 Room Air	RFI*	10:52
03 Room Air	0.000	10:53

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = QWERTYUIOPASDFG
Dr. Lic. = ND/DIS821456
Lot No = 23411080A1
Cyl No = 60
Expiration Date = 10/01/2013
County = 21 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI TEST*