

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004191 Location: VALL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
  - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3375 Lot No. ICS#2 Rep AC 0.100AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 047 Exp. Date 08/01/2014  
Test 1 0.079 Test 4 0.079 Test 7 0.079  
Test 2 0.079 Test 5 0.079 Test 8 0.079  
Test 3 0.080 Test 6 0.079 Test 9 0.080  
Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 867 psi Reg 850 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/6/2014  
Date

Charles E. Eden  
Field Inspector's Signature

07 May 2014 ✓  
Date

Deb Kashur  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004191  
Location = VALL                    8164.13.00 06/09  
05/06/2014                            14:08

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            47  
Standard Expiration?        08/01/2014  
Oper No?                    666666

Flow Cal. Date:            01/28/2010  
Slope                        738  
Intercept                   -777574

IR Calibration Date:        01/28/2010  
                                  3um                    9um

-----  
0th Coef(\*100):            -23128                    -33609  
1st Coef(\*100):            272485                    131724  
2nd Coef(\*100):            1490                      1056  
H2O adj(mg/l\*10k):        432                        614

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINTTEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
05/06/2014      14:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:24
02 Std. Sol.	0.018	14:25
03 Room Air	0.000	14:25
04 Std. Sol.	0.018	14:26
05 Room Air	0.000	14:27
06 Std. Sol.	0.018	14:27
07 Room Air	0.000	14:28

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 02

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
05/06/2014      14:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Sol.	0.291	14:32
03 Room Air	0.000	14:33
04 Std. Sol.	0.293	14:33
05 Room Air	0.000	14:34
06 Std. Sol.	0.295	14:35
07 Room Air	0.000	14:35


08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 02

Oper No. = 666666

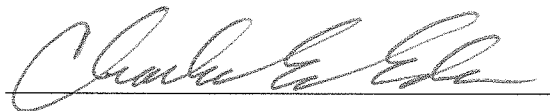

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004191  
Location = VALL                    8164.13.00 06/09  
05/06/2014                            14:14

Test	AC	Time
01 Room Air	0.000	14:15
02 *Subject Test	INT*	14:15
03 Room Air	0.000	14:16

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTEREFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 47  
Expiration Date = 08/01/2014  
County = 02                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *INTERFERENT CHECK*  
*0.100 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
05/06/2014      14:10

Test	AC	Time
01 Diagnostic	OK	14:12
02 Room Air	RFI*	14:12
03 Room Air	0.000	14:12

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 47  
Expiration Date = 08/01/2014  
County = 02      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
05/06/2014      14:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:38
02 Std. Gas	0.079	14:38
03 Room Air	0.000	14:39
04 Std. Gas	0.079	14:39
05 Room Air	0.000	14:40
06 Std. Gas	0.080	14:40
07 Room Air	0.000	14:41

Lot No = 16512080A2  
Cyl No = 47  
Exp Date = 08/01/2014  
County = 02      Oper No. = 666666

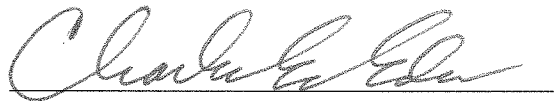

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
05/06/2014      14:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Gas	0.079	14:42
03 Room Air	0.000	14:43
04 Std. Gas	0.079	14:43
05 Room Air	0.000	14:44
06 Std. Gas	0.079	14:44
07 Room Air	0.000	14:45

Lot No = 16512080A2  
Cyl No = 47  
Exp Date = 08/01/2014  
County = 02      Oper No. = 666666

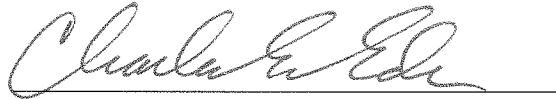

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004191  
Location = VALL      8164.13.00 06/09  
05/06/2014      14:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Gas	0.079	14:46
03 Room Air	0.000	14:46
04 Std. Gas	0.079	14:47
05 Room Air	0.000	14:47
06 Std. Gas	0.080	14:48
07 Room Air	0.000	14:48

Lot No = 16512080A2  
Cyl No = 47  
Exp Date = 08/01/2014  
County = 02      Oper No. = 666666

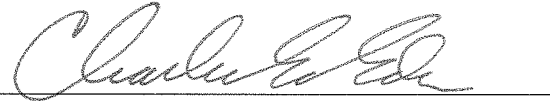

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC