

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004939

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>12210</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>12050</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>—</u> Rep AC <u>0.10 AC + acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. 23411080A1 Cyl No. 15 Exp. Date 10/01/2013

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>

Average 0.080

L. Tank pressure, Level 3 Func D Sub G P: 308 psi Reg 300 psi

M. Remarks/Maintenance record: Annual inspection

06 May 13
Date

Deb Kashur
Field Inspector's Signature

07 MAY 13
Date

Charles E. Eden
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 10:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Sol.	0.019	10:58
03 Room Air	0.000	10:59
04 Std. Sol.	0.019	11:00
05 Room Air	0.000	11:00
06 Std. Sol.	0.019	11:01
07 Room Air	0.000	11:01

08 Sim Temp = 34.0°C

Simul Ser No = DR5132

Std Sol No = 12210

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 11:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:03
02 Std. Sol.	0.296	11:04
03 Room Air	0.000	11:04
04 Std. Sol.	0.296	11:05
05 Room Air	0.000	11:06
06 Std. Sol.	0.296	11:06
07 Room Air	0.000	11:07

08 Sim Temp = 34.0°C

Simul Ser No = DR3803

Std Sol No = 12050

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 11:12

Test	AC	Time
01 Room Air	0.000	11:13
02 *Subject Test	INT*	11:13
03 Room Air	0.000	11:13

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent detected
Form 106-I8000

*0.100 AC
+ acetone*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 11:17

Test	AC	Time
01 Diagnostic	OK	11:17
02 Room Air	RFI*	11:18
03 Room Air	0.000	11:18

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 15
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 09:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:28
02 Std. Gas	0.080	09:29
03 Room Air	0.000	09:29
04 Std. Gas	0.081	09:29
05 Room Air	0.000	09:30
06 Std. Gas	0.080	09:30
07 Room Air	0.000	09:31

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 09:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:58
02 Std. Gas	0.080	09:59
03 Room Air	0.000	09:59
04 Std. Gas	0.080	09:59
05 Room Air	0.000	10:00
06 Std. Gas	0.080	10:00
07 Room Air	0.000	10:01

Lot No = 23411080A1

Cyl No = 15

Exp Date = 10/01/2013

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.13.00 06/09
05/06/2013 10:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:06
02 Std. Gas	0.080	10:07
03 Room Air	0.000	10:07
04 Std. Gas	0.081	10:08
05 Room Air	0.000	10:08
06 Std. Gas	0.080	10:09
07 Room Air	0.000	10:09

Lot No = 23411080A1
Cyl No = 15
Exp Date = 10/01/2013
County = 08 Oper No. = 777777


Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC