

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004952

Location: ADAMS COUNTY (HETT)
S.O.

- Check When Done
- A. Is the warm-up time less than 20 minutes? ✓
 - B. Is a three-pronged grounded outlet used? ✓
 - C. Is the breath tube heated? ✓
 - D. Is the diagnostic check complete? ✓
 - E. Is the time, date, and year correct (re-set if necessary)? ✓
 - F. Print test. (Attach test record.) ✓
 - G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3455 Lot No. 12210 Rep AC 0.020 ✓
 - H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3847 Lot No. 13020 Rep AC 0.300 ✓
 - I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5191 Lot No. ICS #1 Rep AC 0.100 AC + 0.05% ACETONE ✓
 - J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 23411080A1 Cyl No. 59 Exp. Date 10/01/2013

Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>

Average 0.079

L. Tank pressure, Level 3 Func D Sub G P: 911 psi Reg 900 psi

M. Remarks/Maintenance record: ANNUAL INSPECTION

06 MAY 13
Date

Charles E. Eden
Field Inspector's Signature

07 May 13
Date

Deb Kashner
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:48
02 Std. Sol.	0.017	12:48
03 Room Air	0.000	12:49
04 Std. Sol.	0.017	12:50
05 Room Air	0.000	12:50
06 Std. Sol.	0.017	12:51
07 Room Air	0.000	12:51

08 Sim Temp = 34.0°C

Simul Ser No = DR3455
Std Sol No = 12210
County = 01 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC (Sim. Sol.)

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:41
02 Std. Sol.	0.296	12:42
03 Room Air	0.000	12:42
04 Std. Sol.	0.298	12:43
05 Room Air	0.000	12:43
06 Std. Sol.	0.298	12:44
07 Room Air	0.000	12:45

08 Sim Temp = 34.0°C

Simul Ser No = DR3847
Std Sol No = 13020
County = 01 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *LINEARITY TEST*
0.300AC (Sim. Sol.)


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:22

Test	AC	Time
01 Room Air	0.000	12:23
02 *Subject Test	INT*	12:23
03 Room Air	0.000	12:24

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = QWERTYUIOPASDFG
Dr. Lic. = ND/DIS821456
Lot No = 23411080A1
Cyl No = 59
Expiration Date = 10/01/2013
County = 01 Oper No. = 666666

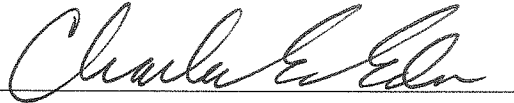

Operator Signature
CHARLES EDER

Remarks: *INTERFERENT CHECK*
0.100AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:25

Test	AC	Time
01 Diagnostic	OK	12:26
02 Room Air	RFI*	12:26
03 Room Air	0.000	12:26

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = MNBVCXZLKJH1234
Dr. Lic. = ND/DIS821456
Lot No = 23411080A1
Cyl No = 59
Expiration Date = 10/01/2013
County = 01 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

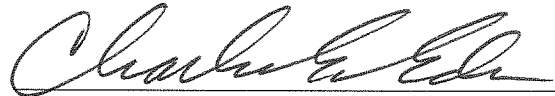

Operator Signature
CHARLES EDER

Remarks: RFI. CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:28
02 Std. Gas	0.078	12:28
03 Room Air	0.000	12:28
04 Std. Gas	0.079	12:29
05 Room Air	0.000	12:29
06 Std. Gas	0.079	12:30
07 Room Air	0.000	12:30

Lot No = 23411080A1
Cyl No = 59
Exp Date = 10/01/2013
County = 01 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC (E.G.S.)


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:31
02 Std. Gas	0.079	12:31
03 Room Air	0.000	12:32
04 Std. Gas	0.079	12:32
05 Room Air	0.000	12:33
06 Std. Gas	0.078	12:33
07 Room Air	0.000	12:34

Lot No = 23411080A1
Cyl No = 59
Exp Date = 10/01/2013
County = 01 Oper No. = 666666

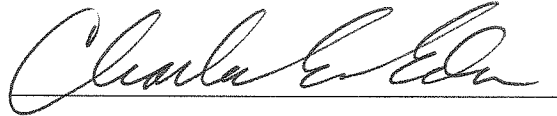

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC (E.G.S.)

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = HETT 8164.13.00 06/09
05/06/2013 12:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:34
02 Std. Gas	0.079	12:35
03 Room Air	0.000	12:35
04 Std. Gas	0.079	12:36
05 Room Air	0.000	12:36
06 Std. Gas	0.079	12:36
07 Room Air	0.000	12:37

Lot No = 23411080A1
Cyl No = 59
Exp Date = 10/01/2013
County = 01 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080AC (E.G.S.)
Form 106-I8000