

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004949

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>10290</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>11110</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>---</u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>23411080A1</u>	Cyl No. <u>49</u>	Exp. Date <u>10/01/2013</u>
Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.080</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 666 psi Reg 660 psi

M. Remarks/Maintenance record: Annual inspection

08 May 2012
Date

Deb Kashur
Field Inspector's Signature

08 MAY 12
Date

Charles E. Edr
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/07/2012 16:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:50
02 Std. Sol.	0.019	16:51
03 Room Air	0.000	16:51
04 Std. Sol.	0.019	16:52
05 Room Air	0.000	16:53
06 Std. Sol.	0.019	16:53
07 Room Air	0.000	16:54

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks: *Low AC*
DKincerity test 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/08/2012 08:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:29
02 Std. Sol.	0.289	08:30
03 Room Air	0.000	08:30
04 Std. Sol.	0.290	08:31
05 Room Air	0.000	08:31
06 Std. Sol.	0.290	08:32
07 Room Air	0.000	08:33

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/08/2012 08:56

Test	AC	Time
01 Room Air	0.000	08:56
02 *Subject Test	INT*	08:57
03 Room Air	0.000	08:57

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 49
Expiration Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.10V AC + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/07/2012 15:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Gas	0.078	15:42
03 Room Air	0.000	15:42
04 Std. Gas	0.079	15:43
05 Room Air	0.000	15:43
06 Std. Gas	0.079	15:44
07 Room Air	0.000	15:44

Lot No = 23411080A1
Cyl No = 49
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/07/2012 15:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:49
02 Std. Gas	0.079	15:49
03 Room Air	0.000	15:50
04 Std. Gas	0.079	15:50
05 Room Air	0.000	15:50
06 Std. Gas	0.079	15:51
07 Room Air	0.000	15:51

Lot No = 23411080A1
Cyl No = 49
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:
calibration check 0.080 Ac

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/07/2012 16:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:46
02 Std. Gas	0.080	16:46
03 Room Air	0.000	16:47
04 Std. Gas	0.080	16:47
05 Room Air	0.000	16:48
06 Std. Gas	0.080	16:48
07 Room Air	0.000	16:48

Lot No = 23411080A1
Cyl No = 49
Exp Date = 10/01/2013
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC