



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number 80-004938	Instrument Location COOP (COOPERSTOWN)
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) INSTALL DURING ANNUAL INSPECTION	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C). ✓
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air). ✓
- 7. Repair and/or Maintenance Performed (if Any): **NA-INSTALL DURING ANNUAL INSP.**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Ed</i>	Date 09 MAY 13
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date 13 May 13

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = COOP 8164.13.00 06/09
05/09/2013 13:58

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 30
Standard Expiration? 08/01/2014
Oper No? 666666

Flow Cal. Date: 01/25/2010
Slope 712
Intercept -604165

IR Calibration Date: 05/02/2013
 3um 9um

0th Coef(*100):	-32801	-32122
1st Coef(*100):	279293	140886
2nd Coef(*100):	-301	554
H2O adj(mg/l*10k):	660	569

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = COOP 8164.13.00 06/09
05/09/2013 14:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:01
02 Std. Gas	0.079	14:02
03 Room Air	0.000	14:02
04 Std. Gas	0.078	14:03
05 Room Air	0.000	14:03
06 Std. Gas	0.079	14:04
07 Room Air	0.000	14:04

Lot No = 16512080A2
Cyl No = 30
Exp Date = 08/01/2014
County = 20 Oper No. = 666666

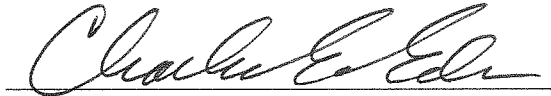

Operator Signature
CHARLES EDER

Remarks: *ACA TEST*
0.080 AC (E.G.S.)

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = COOP 8164.13.00 06/09
05/09/2013 14:09

Test	AC	Time
01 Diagnostic	OK	14:10
02 Room Air	RFI*	14:10
03 Room Air	0.000	14:10

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = QWERTYUIOPASDFG
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 30
Expiration Date = 08/01/2014
County = 20 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI TEST