



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-004935</i>	Instrument Location <i>Ward Co Corr.</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *Install during annual inspect*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Deb Kasher</i>	Date <i>10 May 13</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Edr.</i>	Date <i>13 MAY 13</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edr.* Certified Date: *29 Nov. 2016*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = WARD 8164.13.00 06/09
05/10/2013 09:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOpqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOpqrstuvwxyz!@#\$\$%^&*()_+?


Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 15
Standard Expiration? 08/01/2014
Oper No? 777777

Flow Cal. Date: 07/12/2012
Slope 655
Intercept -510289

IR Calibration Date: 04/23/2013
 3um 9um


0th Coef(*100): -11131 -13952
1st Coef(*100): 263852 129376
2nd Coef(*100): 3246 1683
H2O adj(mg/l*10k): 464 535

***** Printer Test End *****



Operator Signature
DEB KASHUR

Remarks:

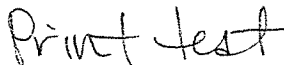

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = WARD 8164.13.00 06/09
05/10/2013 09:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:54
02 Std. Gas	0.079	09:55
03 Room Air	0.000	09:55
04 Std. Gas	0.078	09:55
05 Room Air	0.000	09:56
06 Std. Gas	0.080	09:56
07 Room Air	0.000	09:57

Lot No = 16512080A2
Cyl No = 15
Exp Date = 08/01/2014
County = 51 Oper No. = 777777

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:
Calibration sheet 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = WARD 8164.13.00 06/09
05/10/2013 09:58

Test	AC	Time
01 Diagnostic	OK	09:58
02 Room Air	RFI*	09:58
03 Room Air	0.000	09:59

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 15
Expiration Date = 08/01/2014
County = 51 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000