



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-004939</i>	Instrument Location <i>Renville Co SD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) <i>install during annual inspection</i>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *install*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- N/A* 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Deb Kashner</i>	Date <i>10 May 13</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Eder</i>	Date <i>13 MAY 13</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eder* Certified Date: *23 Nov. 2016*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MOHL 8164.13.00 06/09
05/10/2013 13:24

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPNORSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPNORSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 8
Standard Expiration? 08/01/2014
Oper No? 777777

Flow Cal. Date: 01/18/2010
Slope 724
Intercept -779308

IR Calibration Date: 03/10/2010
 3um 9um


0th Coef(*100): -45908 -35439
1st Coef(*100): 263128 135479
2nd Coef(*100): 1658 1038
H2O adj(mg/l*10k): 832 527

***** Printer Test End *****



Operator Signature
DEB KASHUR

Remarks:

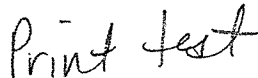

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MOHL 8164.13.00 06/09
05/10/2013 13:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:28
02 Std. Gas	0.080	13:28
03 Room Air	0.000	13:29
04 Std. Gas	0.080	13:29
05 Room Air	0.000	13:30
06 Std. Gas	0.081	13:30
07 Room Air	0.000	13:30

Lot No = 16512080A2
Cyl No = 8
Exp Date = 08/01/2014
County = 38 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = MOHL 8164.13.00 06/09
05/10/2013 13:31

Test	AC	Time
01 Diagnostic	OK	13:31
02 Room Air	RFI*	13:32
03 Room Air	0.000	13:32

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 8
Expiration Date = 08/01/2014
County = 38 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000