



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Deb Shanaver</i>		
Location <i>Cavalier Co SO - Langdon</i>	Intoxilyzer® Serial Number <i>80-004938</i>	
Gas Lot Number <i>19615080 AS</i>	Gas Cylinder Number <i>15</i>	Gas Expiration Date <i>09/05/2017</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 079 AC
 - B. 0. 079 AC
 - C. 0. 080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Deb Shanaver</i>	Date <i>10 May 16 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Edr</i>	Date <i>12 MAY 16</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.13.00 06/09
05/10/2016 16:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:34
02 Std. Gas	0.079	16:35
03 Room Air	0.000	16:35
04 Std. Gas	0.079	16:36
05 Room Air	0.000	16:36
06 Std. Gas	0.080	16:36
07 Room Air	0.000	16:37

Lot No = 19615080A5
Cyl No = 15
Exp Date = 09/05/2017
County = 10 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks: ACA
 Calibration check 0.080AC