



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-006497</i>	Instrument Location <i>Grand Forks AFB, ND</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NA*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Raymond Chiavetta</i>	Date <i>10 MAY 16 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>08 Aug 16</i>




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006497  
Location = GFAF      8164.13.00 06/09  
05/10/2016      13:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:36
02 Std. Gas	0.077	13:36
03 Room Air	0.000	13:37
04 Std. Gas	0.077	13:37
05 Room Air	0.000	13:38
06 Std. Gas	0.078	13:39
07 Room Air	0.000	13:39

Lot No = 19615080A5  
Cyl No = 30  
Exp Date = 09/05/2017  
County = 65      Oper No. = 130053

  
\_\_\_\_\_  
Operator Signature  
RAYMOND CHIAVETTA

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006497  
Location = GFAF      8164.13.00 06/09  
05/10/2016      13:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	13:59
02 Room Air	RFI*	13:59

\*Invalid Test  
Inhibited - RFI

Lot No = 19615080A5  
Cyl No = 30  
Exp Date = 09/05/2017  
County = 65      Oper No. = 130053



Operator Signature  
RAYMOND CHIAVETTA

Remarks:

Form 106-I8000