



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-006489</i>	Instrument Location <i>GFPD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *New Installation*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Don Langley</i>	Date <i>05-12-16 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>DS 16 May 2016</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006489  
Location = GFPD                    8164.13.00 06/09  
05/12/2016                            12:00

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

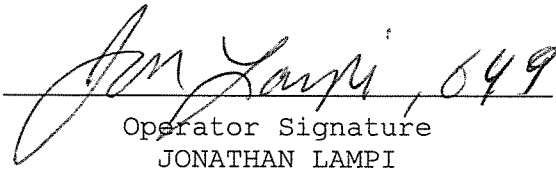
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?           1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            01316080A2  
Standard Cyl #?            26  
Standard Expiration?       02/05/2018  
Oper No?                    039947

Flow Cal. Date:            08/18/2015  
Slope                       663  
Intercept                  -631787

IR Calibration Date:       08/18/2015  
                                        3um                    9um

-----  
0th Coef(\*100):           -24626                    -16551  
1st Coef(\*100):           269218                    130789  
2nd Coef(\*100):           3390                      1656  
H2O adj(mg/l\*10k):       742                       460

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
JONATHAN LAMPI

Remarks:

Print Test for Install of new machine

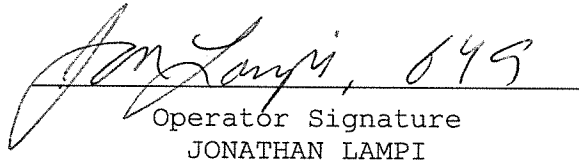
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.13.00 06/09  
05/12/2016      12:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:02
02 Std. Gas	0.080	12:02
03 Room Air	0.000	12:03
04 Std. Gas	0.080	12:03
05 Room Air	0.000	12:04
06 Std. Gas	0.080	12:04
07 Room Air	0.000	12:05

Lot No = 01316080A2  
Cyl No = 26  
Exp Date = 02/05/2018  
County = 18      Oper No. = 039947

  
Operator Signature  
JONATHAN LAMPI

Remarks:

Form 106-I8000

ACA test for Install

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.13.00 06/09  
05/12/2016      12:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	12:06
02 Room Air	0.000	12:07

\*Invalid Test  
Inhibited - RFI

Lot No = 01316080A2  
Cyl No = 26  
Exp Date = 02/05/2018  
County = 18      Oper No. = 039947



Operator Signature  
JONATHAN LAMPI

Remarks:

Form 106-I8000

RFI test for Install