

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004943

Location: Minot Air Force Base

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5191 Lot No. 15160 Rep AC 6.620 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3847 Lot No. 16060 Rep AC 0.360 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5190 Lot No. ICS4 Solution 0.10 Acet 6.05%<sup>m</sup> ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test. acetone  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 01316080A2 Cyl No. 30 Exp. Date 02/05/2018

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 993 psi Reg ~1/500 psi

K. Remarks/Maintenance record: Annual inspection, replaced o-rings on breath hose & simulator return

11 May 2016  
Date

12 MAY 2016 ✓  
Date

Deb Shanaver  
Field Inspector's Signature

Charles E. Eder  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004943  
Location = MNAF                    8164.13.00 06/09  
05/11/2016                            13:40

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^&\*()\_+?

Current Instrument Setup

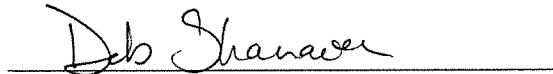
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        No  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            01316080A2  
Standard Cyl #?            30  
Standard Expiration?        02/05/2018  
Oper No?                    777777

Flow Cal. Date:            01/18/2010  
Slope                        693  
Intercept                   -665387

IR Calibration Date:        05/21/2015  
                                  3um                    9um

0th Coef(*100):	-28195	-26668
1st Coef(*100):	268915	137995
2nd Coef(*100):	103	992
H2O adj(mg/l*10k):	222	308

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004943  
Location = MNAF                    8164.13.00 06/09  
05/11/2016                            14:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Sol.	0.017	14:05
03 Room Air	0.000	14:06
04 Std. Sol.	0.017	14:07
05 Room Air	0.000	14:07
06 Std. Sol.	0.017	14:08
07 Room Air	0.000	14:08

08 Sim Temp = 34.0°C

Simul Ser No = DR5191  
Std Sol No = 15160  
County = 64                            Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

*low alcohol      0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004943  
Location = MNAF      8164.13.00 06/09  
05/11/2016      14:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Sol.	0.293	14:11
03 Room Air	0.000	14:12
04 Std. Sol.	0.296	14:13
05 Room Air	0.000	14:13
06 Std. Sol.	0.299	14:14
07 Room Air	0.000	14:14

08 Sim Temp = 34.0°C

Simul Ser No = DR3847  
Std Sol No = 16060  
County = 64      Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:  
High alcohol      0.300 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004943  
Location = MNAF                8164.13.00 06/09  
05/11/2016                        14:16

Test	AC	Time
01 Room Air	0.000	14:17
02 *Subject Test	INT*	14:18
03 Room Air	0.000	14:18

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 01316080A2  
Cyl No = 30  
Expiration Date = 02/05/2018  
County = 64                          Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected 0.10AC + 0.05%  
acetone*

Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004943  
Location = MNAF                8164.13.00 06/09  
05/11/2016                        14:18

Test	AC	Time
01 Diagnostic	OK	14:19
02 Room Air	RFI*	14:19
03 Room Air	0.000	14:20

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 01316080A2  
Cyl No = 30  
Expiration Date = 02/05/2018  
County = 64                        Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

RFI test

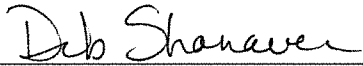
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004943  
Location = MNAF                    8164.13.00 06/09  
05/11/2016                            13:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:53
02 Std. Gas	0.079	13:53
03 Room Air	0.000	13:54
04 Std. Gas	0.079	13:54
05 Room Air	0.000	13:54
06 Std. Gas	0.079	13:55
07 Room Air	0.000	13:55

Lot No = 01316080A2  
Cyl No = 30  
Exp Date = 02/05/2018  
County = 64                            Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:  
*Calibration check 0.080AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004943  
Location = MNAF      8164.13.00 06/09  
05/11/2016      13:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:56
02 Std. Gas	0.079	13:56
03 Room Air	0.000	13:57
04 Std. Gas	0.079	13:57
05 Room Air	0.000	13:58
06 Std. Gas	0.078	13:58
07 Room Air	0.000	13:59

Lot No = 01316080A2  
Cyl No = 30  
Exp Date = 02/05/2018  
County = 64      Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:  
*Calibration check 0.080AC*



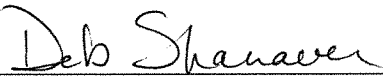
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004943  
Location = MNAF                8164.13.00 06/09  
05/11/2016                        13:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:59
02 Std. Gas	0.080	14:00
03 Room Air	0.000	14:00
04 Std. Gas	0.079	14:01
05 Room Air	0.000	14:01
06 Std. Gas	0.079	14:01
07 Room Air	0.000	14:02

Lot No = 01316080A2  
Cyl No = 30  
Exp Date = 02/05/2018  
County = 64                      Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB SHANAVER

Remarks:

*Calibration check 0.080AC*