

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004945

Location: McIntosh Co ^{DK} SO

- Check When Done
- A. Is the warm-up time less than 20 minutes? ✓
 - B. Is a three-pronged grounded outlet used? ✓
 - C. Is the breath tube heated? ✓
 - D. Is the diagnostic check complete? ✓
 - E. Is the time, date, and year correct (re-set if necessary)? ✓
 - F. Print test. (Attach test record.) ✓
 - G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5132 Lot No. 12010 Rep AC 0.020 ✓
 - H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3803 Lot No. 12050 Rep AC 0.300 ✓
 - I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR3846 Lot No. — Rep AC 0.100 AC + acetone ✓
 - J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>23411080A1</u>	Cyl No. <u>48</u>	Exp. Date <u>10/01/2013</u>
Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 1006 psi Reg 1000 psi

M. Remarks/Maintenance record: Annual inspections

15 May 2012
Date

Deb Kasher
Field Inspector's Signature

17 MAY 12
Date

Charles E. Elder
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 13:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:16
02 Std. Sol.	0.016	13:17
03 Room Air	0.000	13:17
04 Std. Sol.	0.016	13:18
05 Room Air	0.000	13:19
06 Std. Sol.	0.017	13:19
07 Room Air	0.000	13:20

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 26

Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 13:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:22
02 Std. Sol.	0.298	13:23
03 Room Air	0.000	13:23
04 Std. Sol.	0.297	13:24
05 Room Air	0.000	13:25
06 Std. Sol.	0.298	13:25
07 Room Air	0.000	13:26

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 26

Oper No. = 777777

Deb Keshur
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 13:34

Test	AC	Time
01 Room Air	0.000	13:35
02 *Subject Test	INT*	13:36
03 Room Air	0.000	13:36

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 48
Expiration Date = 10/01/2013
County = 26 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 13:39

Test	AC	Time
01 Diagnostic	OK	13:39
02 Room Air	RFI*	13:40
03 Room Air	0.000	13:40

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 48
Expiration Date = 10/01/2013
County = 26 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 12:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:00
02 Std. Gas	0.079	13:00
03 Room Air	0.000	13:00
04 Std. Gas	0.080	13:01
05 Room Air	0.000	13:01
06 Std. Gas	0.080	13:02
07 Room Air	0.000	13:02

Lot No = 23411080A1
Cyl No = 48
Exp Date = 10/01/2013
County = 26 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 Ac


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 13:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:04
02 Std. Gas	0.080	13:04
03 Room Air	0.000	13:05
04 Std. Gas	0.080	13:05
05 Room Air	0.000	13:05
06 Std. Gas	0.080	13:06
07 Room Air	0.000	13:06

Lot No = 23411080A1
Cyl No = 48
Exp Date = 10/01/2013
County = 26 Oper No. = 777777

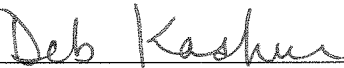

Operator Signature
DEB KASHUR

Remarks:
Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = ASHL 8164.13.00 06/09
05/15/2012 13:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:07
02 Std. Gas	0.080	13:08
03 Room Air	0.000	13:08
04 Std. Gas	0.080	13:08
05 Room Air	0.000	13:09
06 Std. Gas	0.079	13:09
07 Room Air	0.000	13:10

Lot No = 23411080A1
Cyl No = 48
Exp Date = 10/01/2013
County = 26 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

calibration check 0.080AC