



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <b>80-004944</b>	Instrument Location <b>WFPD (WEST FARGO)</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <b>INSTALL DURING ANNUAL INSPECTION</b>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P). ✓
  - B. ACA Test (Level 1, Function C). ✓
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air). ✓
- 7. Repair and/or Maintenance Performed (if Any): **INSTALL DURING ANNUAL INSPECTION**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Egan</i>	Date <b>15 MAY 13</b>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <b>16 May 13</b>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004944  
Location = WFPD                8164.13.00 06/09  
05/15/2013                      15:41

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            18  
Standard Expiration?       08/01/2014  
Oper No?                    666666

Flow Cal. Date:            07/01/2011  
Slope                      685  
Intercept                  -634740

IR Calibration Date:        07/01/2011  
                                    3um                      9um

-----  
0th Coef(\*100):            -22591                    -17234  
1st Coef(\*100):            265915                    132691  
2nd Coef(\*100):            2841                      1562  
H2O adj(mg/l\*10k):        559                        425

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004944  
Location = WFPD      8164.13.00 06/09  
05/15/2013      15:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Gas	0.081	15:42
03 Room Air	0.000	15:43
04 Std. Gas	0.081	15:43
05 Room Air	0.000	15:44
06 Std. Gas	0.081	15:44
07 Room Air	0.000	15:45

Lot No = 16512080A2  
Cyl No = 18  
Exp Date = 08/01/2014  
County = 09      Oper No. = 666666

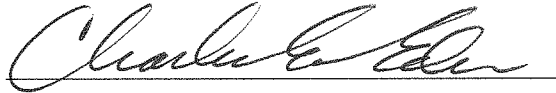

Operator Signature  
CHARLES EDER

Remarks: *ACA TEST*  
*0.080 AC (E.G.S.)*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004944  
Location = WFPD                    8164.13.00 06/09  
05/15/2013                            15:46

Test	AC	Time
01 Diagnostic	OK	15:47
02 Room Air	RFI*	15:47
03 Room Air	0.000	15:47

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = QWERTYUIOPASDFG  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 18  
Expiration Date = 08/01/2014  
County = 09                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI TEST*