

INTOXILYZER 8000 INSPECTION

Serial No.: 80-002667

Location: Valley City PD

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>100-DR</u> Rep AC <u>0.020</u>
<u>12010</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>12050</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3844</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. <u>11611080A2</u>	Cyl No. <u>133</u>	Exp. Date <u>04/01/2013</u>
Test 1 <u>0.081</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.081</u>		

L. Tank pressure, Level 3 Func D Sub G P: 673 psi Reg 670 psi

M. Remarks/Maintenance record: Annual inspection

16 May 2012
Date

Deb Kashur
Field Inspector's Signature

17 MAY 12
Date

[Signature]
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:25

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 11611080A2
Standard Cyl #? 133
Standard Expiration? 06/01/2013
Oper No? 777777

Flow Cal. Date: 09/10/2007
Slope 693
Intercept -570251

IR Calibration Date: 06/03/2011
 3um 9um

0th Coef(*100): -3044 -42
1st Coef(*100): 248787 127659
2nd Coef(*100): 6023 1886
H2O adj(mg/l*10k): 632 468

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:40
02 Std. Sol.	0.021	15:40
03 Room Air	0.000	15:41
04 Std. Sol.	0.021	15:42
05 Room Air	0.000	15:42
06 Std. Sol.	0.021	15:43
07 Room Air	0.000	15:43

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 02

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:45
02 Std. Sol.	0.313	15:46
03 Room Air	0.000	15:46
04 Std. Sol.	0.312	15:47
05 Room Air	0.000	15:47
06 Std. Sol.	0.310	15:48
07 Room Air	0.000	15:49

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 02

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:56

Test	AC	Time
01 Room Air	0.000	15:56
02 *Subject Test	INT*	15:57
03 Room Air	0.000	15:57

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 11611080A2
Cyl No = 133
Expiration Date = 06/01/2013
County = 02 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 Ac + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:58

Test	AC	Time
01 Diagnostic	OK	15:58
02 Room Air	RFI*	15:58
03 Room Air	0.000	15:59

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 11611080A2
Cyl No = 133
Expiration Date = 06/01/2013
County = 02 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:28
02 Std. Gas	0.081	15:29
03 Room Air	0.000	15:29
04 Std. Gas	0.080	15:30
05 Room Air	0.000	15:30
06 Std. Gas	0.081	15:30
07 Room Air	0.000	15:31

Lot No = 11611080A2
Cyl No = 133
Exp Date = 06/01/2013
County = 02 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:32
02 Std. Gas	0.080	15:32
03 Room Air	0.000	15:33
04 Std. Gas	0.081	15:33
05 Room Air	0.000	15:33
06 Std. Gas	0.080	15:34
07 Room Air	0.000	15:34

Lot No = 11611080A2

Cyl No = 133

Exp Date = 06/01/2013

County = 02

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = VALL 8164.13.00 06/09
05/16/2012 15:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:35
02 Std. Gas	0.081	15:35
03 Room Air	0.000	15:36
04 Std. Gas	0.081	15:36
05 Room Air	0.000	15:37
06 Std. Gas	0.080	15:37
07 Room Air	0.000	15:38

Lot No = 11611080A2
Cyl No = 133
Exp Date = 06/01/2013
County = 02 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000