

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004948

Location: Studsman Corr (STUT)

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5132</u> Lot No. <u>12010</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>12050</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>—</u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>23411080A1</u>	Cyl No. <u>216</u>	Exp. Date <u>10/01/2013</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Tank pressure, Level 3 Func D Sub G P: 956 psi Reg 960 psi

M. Remarks/Maintenance record: annual inspection

116 May 2012
Date
17 MAY 12
Date

Deb Kashur
Field Inspector's Signature
Charles E. Eden
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 16:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 26
Standard Expiration? 10/01/2013
Oper No? 777777

Flow Cal. Date: 07/14/2010
Slope 778
Intercept -795769

IR Calibration Date: 07/14/2010
 3um 9um

0th Coef(*100): -31061 -21991
1st Coef(*100): 265097 133596
2nd Coef(*100): 802 1052
H2O adj(mg/l*10k): 675 417

***** Printer Test End *****



Operator Signature
DEB KASHUR

Remarks:



Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 16:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:59
02 Std. Sol.	0.019	17:00
03 Room Air	0.000	17:00
04 Std. Sol.	0.018	17:01
05 Room Air	0.000	17:02
06 Std. Sol.	0.019	17:02
07 Room Air	0.000	17:03

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 47

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC ^{DK} ~~0.08~~ 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 17:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:04
02 Std. Sol.	0.302	17:05
03 Room Air	0.000	17:06
04 Std. Sol.	0.302	17:06
05 Room Air	0.000	17:07
06 Std. Sol.	0.300	17:08
07 Room Air	0.000	17:08

08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 47

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 17:10

Test	AC	Time
01 Room Air	0.000	17:11
02 *Subject Test	INT*	17:11
03 Room Air	0.000	17:12

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 26
Expiration Date = 10/01/2013
County = 47 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100 AC + acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 17:12

Test	AC	Time
01 Diagnostic	OK	17:13
02 Room Air	RFI*	17:13
03 Room Air	0.000	17:14

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 26
Expiration Date = 10/01/2013
County = 47 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 16:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:45
02 Std. Gas	0.080	16:45
03 Room Air	0.000	16:45
04 Std. Gas	0.080	16:46
05 Room Air	0.000	16:46
06 Std. Gas	0.080	16:47
07 Room Air	0.000	16:47

Lot No = 23411080A1
Cyl No = 26
Exp Date = 10/01/2013
County = 47 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:
Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 16:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:48
02 Std. Gas	0.080	16:48
03 Room Air	0.000	16:49
04 Std. Gas	0.081	16:49
05 Room Air	0.000	16:49
06 Std. Gas	0.080	16:50
07 Room Air	0.000	16:50

Lot No = 23411080A1

Cyl No = 26

Exp Date = 10/01/2013

County = 47

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = STUT 8164.13.00 06/09
05/16/2012 16:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:52
02 Std. Gas	0.081	16:52
03 Room Air	0.000	16:53
04 Std. Gas	0.080	16:53
05 Room Air	0.000	16:54
06 Std. Gas	0.080	16:54
07 Room Air	0.000	16:54

Lot No = 23411080A1
Cyl No = 26
Exp Date = 10/01/2013
County = 47 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC