



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (03-2014)

Serial Number 80-004943	Instrument Location McKenzie County Sheriff's Office, Watford City, ND
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000. *N/A*
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Dude B. Newman</i>	Date <i>05/16/2014</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>22 May 14</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. [Signature]* Certified Date: *23 Nov. 2016*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = WATF 8164.13.00 06/09
05/16/2014 16:54

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 62
Standard Expiration? 08/01/2014
Oper No? 030924

Flow Cal. Date: 01/18/2010
Slope 693
Intercept -665387

IR Calibration Date: 05/07/2013
3um 9um

	3um	9um
0th Coef(*100):	-10324	-14602
1st Coef(*100):	231940	125849
2nd Coef(*100):	6794	2184
H2O adj(mg/l*10k):	586	616

***** Printer Test End *****

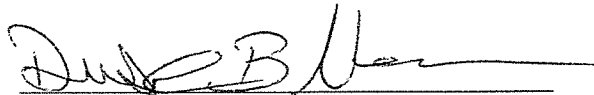


Operator Signature
DUSTIN NEWMAN

Remarks: Good Print Test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = WATF 8164.13.00 06/09
05/16/2014 16:54

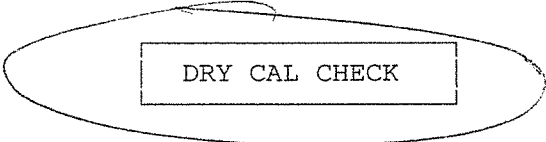

Test	AC	Time
01 Room Air	0.000	16:55
02 Std. Gas	0.079	16:55
03 Room Air	0.000	16:56
04 Std. Gas	0.080	16:56
05 Room Air	0.000	16:57
06 Std. Gas	0.079	16:57
07 Room Air	0.000	16:57

Lot No = 16512080A2
Cyl No = 62
Exp Date = 08/01/2014
County = 27 Oper No. = 030924

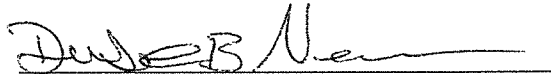

Operator Signature
DUSTIN NEWMAN

Remarks: Good ACA Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = WATF 8164.13.00 06/09
05/16/2014 16:58

Test	AC	Time
01 Room Air	RFI*	16:59
02 Room Air	RFI*	17:00

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 03/25/1969
Sub Sex = Male Weight = 180
Test = DUI Cit = 000000001
Dr. Lic. = MT/03251969111
Lot No = 16512080A2
Cyl No = 62
Expiration Date = 08/01/2014
County = 27 Oper No. = 030924



Operator Signature
DUSTIN NEWMAN

Remarks: Good RFI Test