



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-005945</i>	Instrument Location <i>TAT - New Town</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) <u><i>annual inspection install</i></u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- N/A* 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Deb Kashur</i>	Date <i>16 May 14</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Edr</i>	Date <i>19 MAY 14</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edr* Certified Date: *17 Nov. 2016*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = NEWT 8164.13.00 06/09
05/16/2014 13:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:35
02 Std. Gas	0.078	13:35
03 Room Air	0.000	13:36
04 Std. Gas	0.078	13:36
05 Room Air	0.000	13:36
06 Std. Gas	0.079	13:37
07 Room Air	0.000	13:37

Lot No = 33913080A4
Cyl No = 23
Exp Date = 01/01/2016
County = 60 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks: Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = NEWT 8164.13.00 06/09
05/16/2014 13:38

Test	AC	Time
01 Diagnostic	OK	13:39
02 Room Air	RFI*	13:39
03 Room Air	0.000	13:40

*Invalid Test
 Inhibited - RFI

Sub Name = RFI, NA NA
Sub DOB = 08/01/1975
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 33913080A4
Cyl No = 23
Expiration Date = 01/01/2016
County = 60 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000