



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (5-2011)

Serial Number 80-003061	Instrument Location Devils Lake LEC
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): None Needed
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 5/10/2011
Reviewed By (Crime Laboratory Use Only) Deb Kashner	Date 23 May 2011

80-003061
P. 131 of 135
JM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = DEVL DK 8164.13.00 06/09
05/18/2011 23 May 11 16:31

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 17
Standard Expiration? 07/01/2012
Oper No? 069108

Flow Cal. Date: 03/04/2011
Slope 700
Intercept -623726

IR Calibration Date: 04/26/2011
3um 9um

0th Coef(*100): -10139 -4793
1st Coef(*100): 265984 136608
2nd Coef(*100): 2678 1277
H2O adj(mg/l*10k): 524 433

***** Printer Test End *****

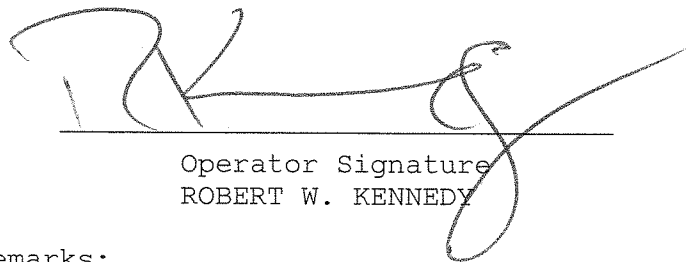
Operator Signature
ROBERT W. KENNEDY

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = ~~DEVL~~ ^{DF} 8164.13.00 06/09
05/18/2011 ^{29 May 11} 16:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:33
02 Std. Gas	0.080	16:33
03 Room Air	0.000	16:33
04 Std. Gas	0.081	16:34
05 Room Air	0.000	16:34
06 Std. Gas	0.081	16:35
07 Room Air	0.000	16:35

Lot No = 15410080A1
Cyl No = 17
Exp Date = 07/01/2012
County = 36 Oper No. = 069108

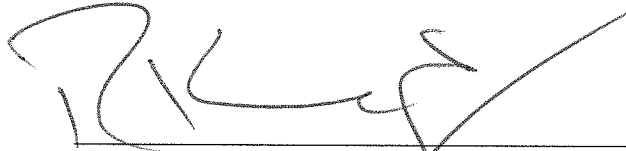

Operator Signature
ROBERT W. KENNEDY

Remarks:

Form 106-I8000

80-003061
P.133 of 135
FM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = *DEVL* 8164.13.00 06/09
05/18/2011 16:42
DK
23 May 11

Test	AC	Time
01 Room Air	RFI*	16:43
02 Room Air	RFI*	16:43

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 09/09/1990
Sub Sex = Male Weight = 222
Test = DUI Cit = 786559
Dr. Lic. = ND/TES907654
Lot No = 15410080A1
Cyl No = 17
Expiration Date = 07/01/2012
County = 36 Oper No. = 069108



Operator Signature
ROBERT KENNEDY

Remarks:

Form 106-I8000

80-003061
P. 134 of 135
FKL

23 May 2011

I received installation paperwork for 80-003061 completed by Robert Kennedy. I emailed Robert and Joe Knowski stating that Robert forgot to enter a location code so the test records print out blank when it should be DEVL. Joe emailed me back stating he will enter the location code.

Deb Kashur

80-003061
P. 135 of 135
YAM